

**Dentium Instruments**  
for Total Solution


Catalog & Manual

# Index

Guide System	<b>Digital Guide</b>	04
	Full Kit	
	Simple Kit	
	<b>Polymer Guide</b>	12
	<b>Implant Guide</b>	14
Metal Kit	<b>Sinus Kit</b>	18
	<b>RE Kit</b>	21
Sinus Instruments	<b>DASK</b>	23
	<b>Osteotome Kit</b>	28
	<b>Sinus Elevator</b>	29
GBR Instruments	<b>RS Kit</b>	31
	<b>Harvest Drill</b>	36
Others	<b>Help Kit</b>	39
	<b>Temporary Shell</b>	46
	<b>White Seal</b>	48
	<b>TN-Brush</b>	52

# Guide System



*Digital Guide*  
Full kit & Simple Kit  
*Polymer Guide*  
*Implant Guide*

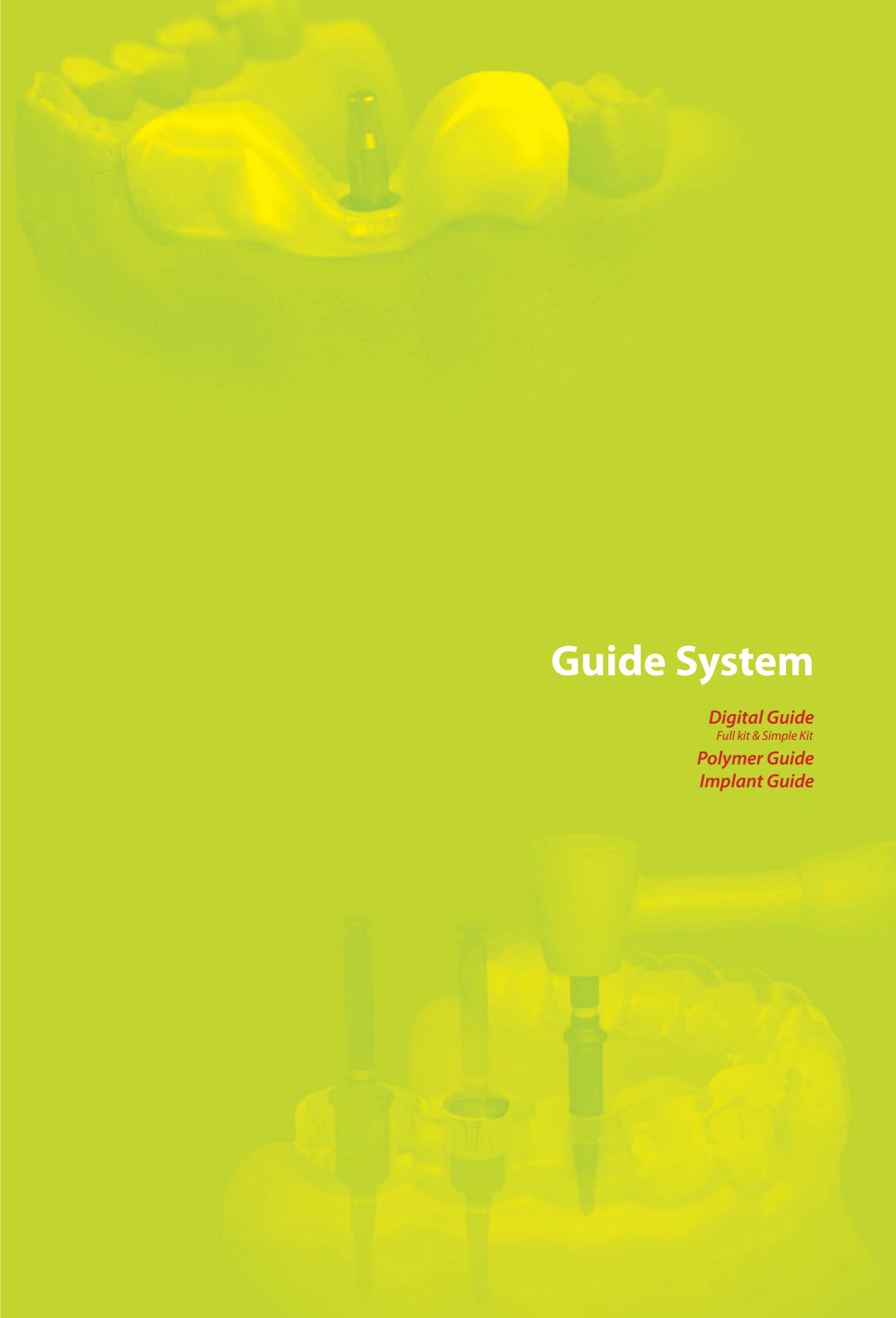

# Digital Guide

Simple & accurate digital guided surgery

- Simple Kit or Full kit
- Non-Sleeve, Open Sleeve or Ti-Sleeve
- Enhanced guided function by step drill

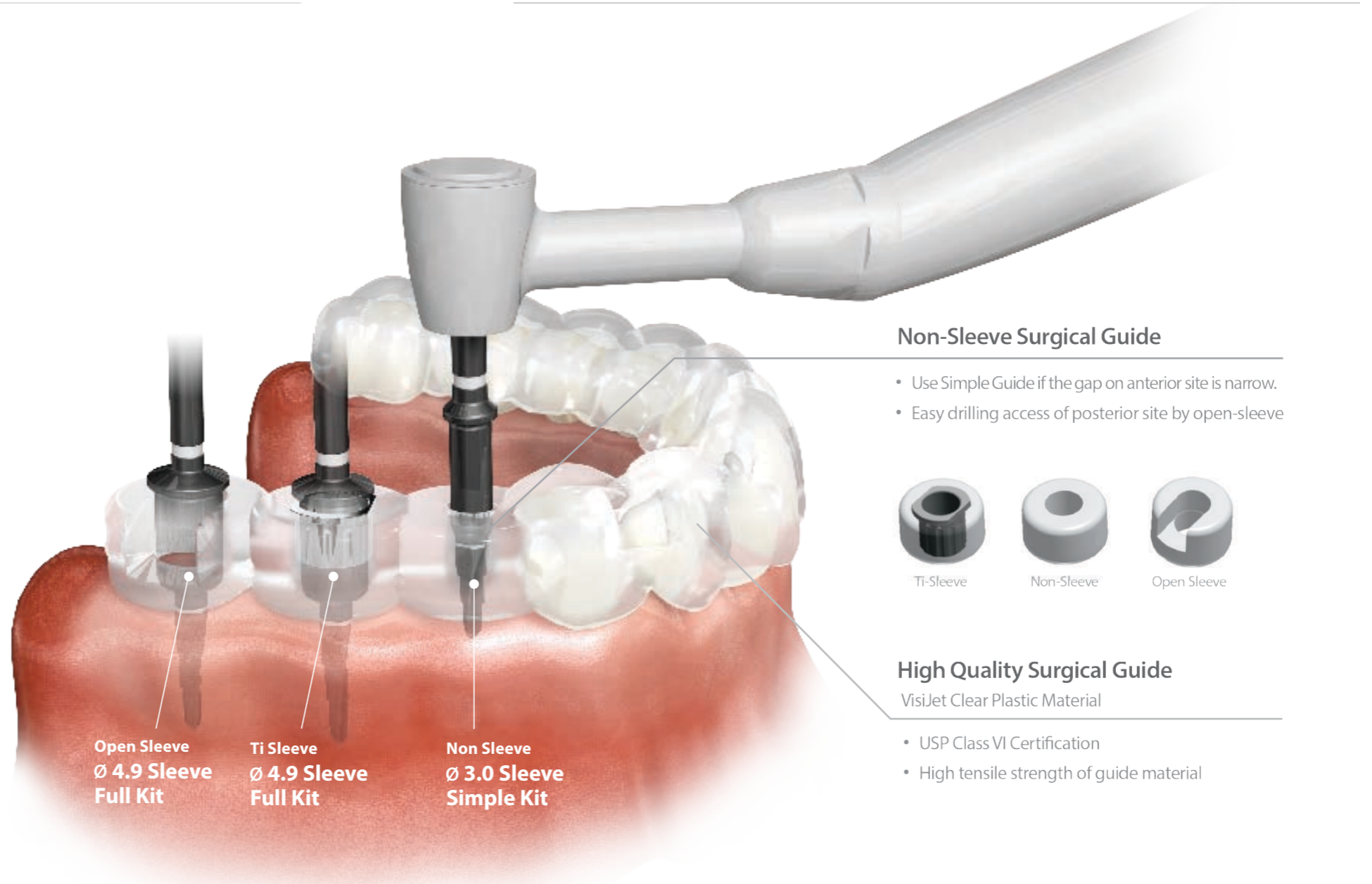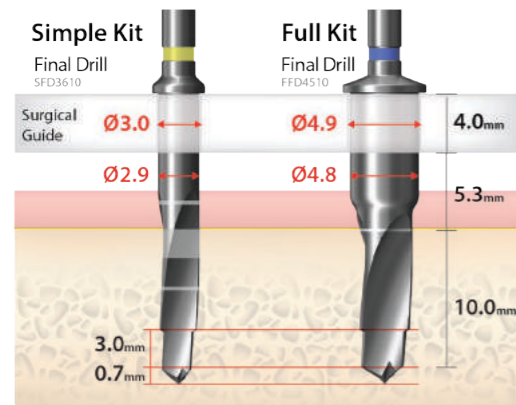

## Non-Sleeve Surgical Guide



- Use Simple Guide if the gap on anterior site is narrow.
- Easy drilling access of posterior site by open-sleeve



## High Quality Surgical Guide

VisiJet Clear Plastic Material



- USP Class VI Certification
- High tensile strength of guide material

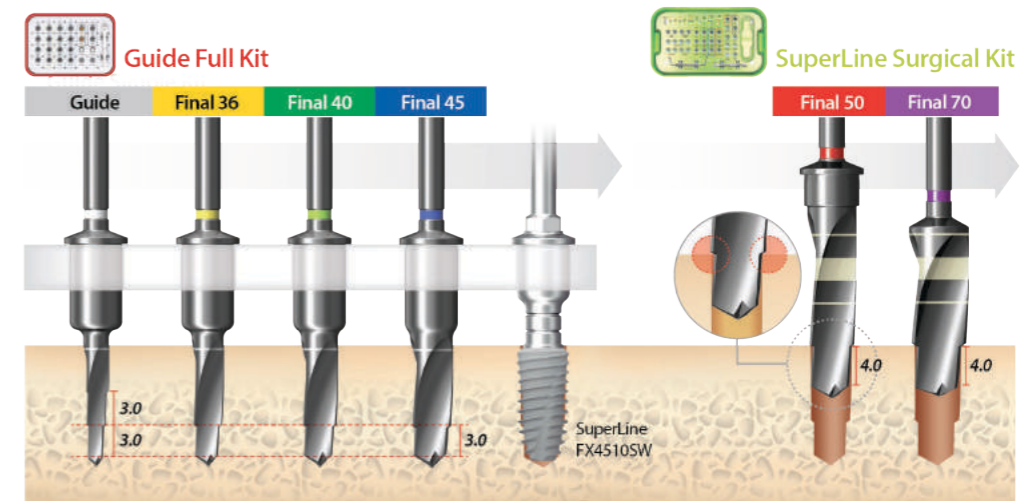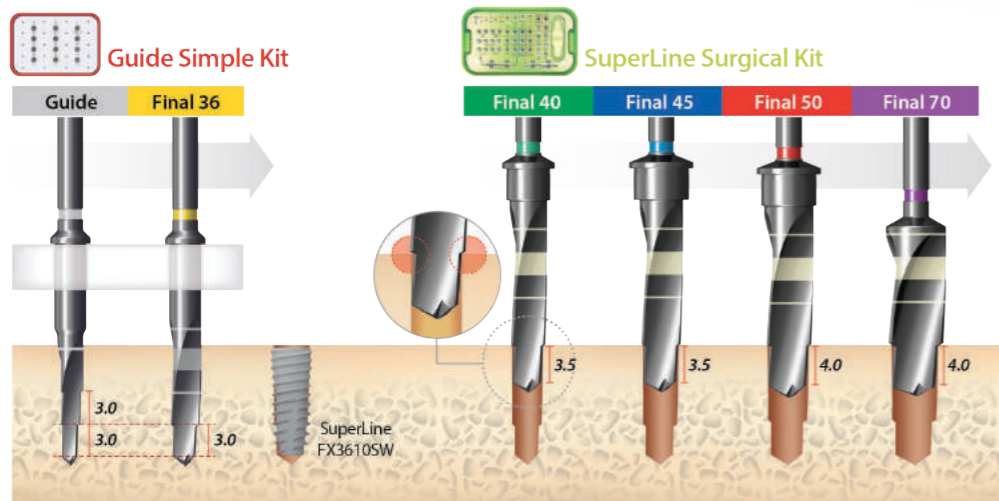



# Full Kit



- For Ø36, Ø40, Ø45 Fixture
- Hybrid design for 6mm Final drill & Countersink

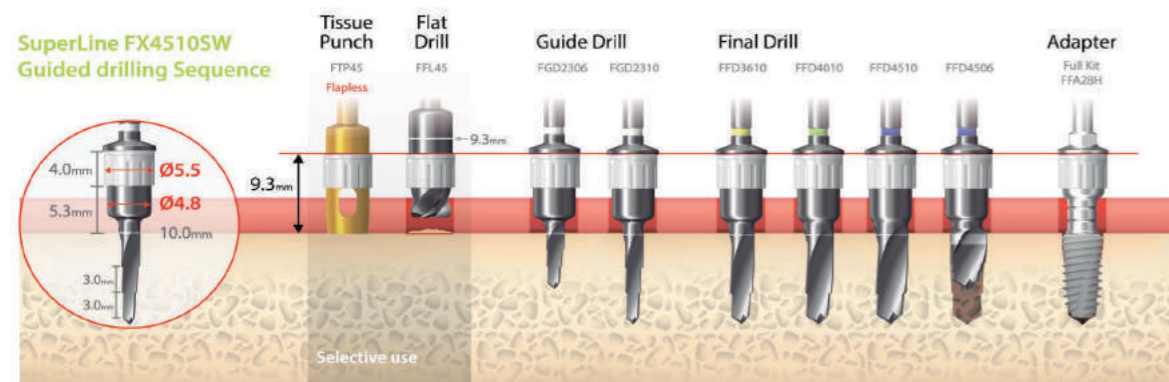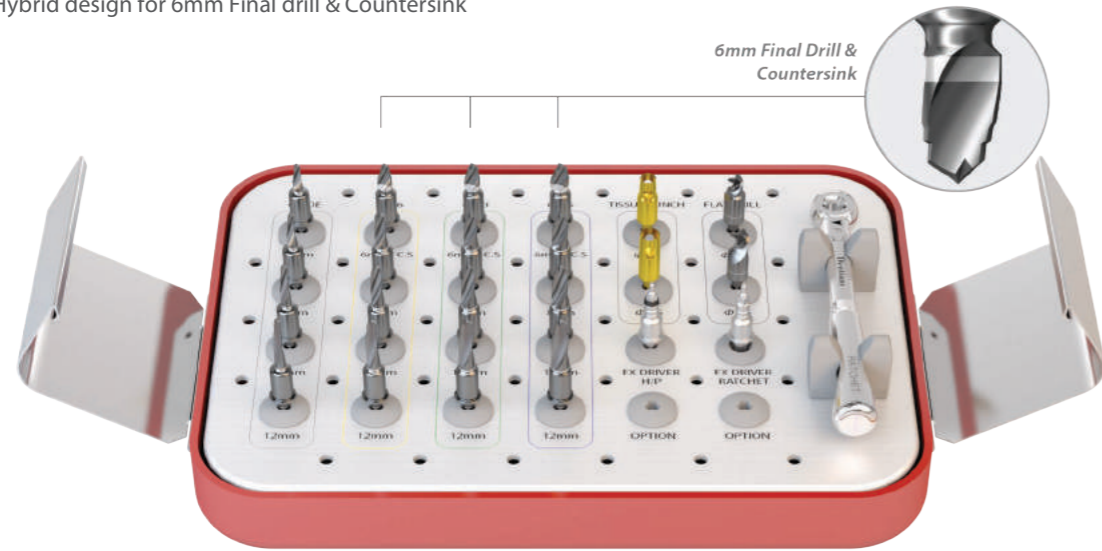

# Simple Kit

*Path&Depth Guided*

- For Ø36 Fixture
- Ø3.0 Narrow Sleeve

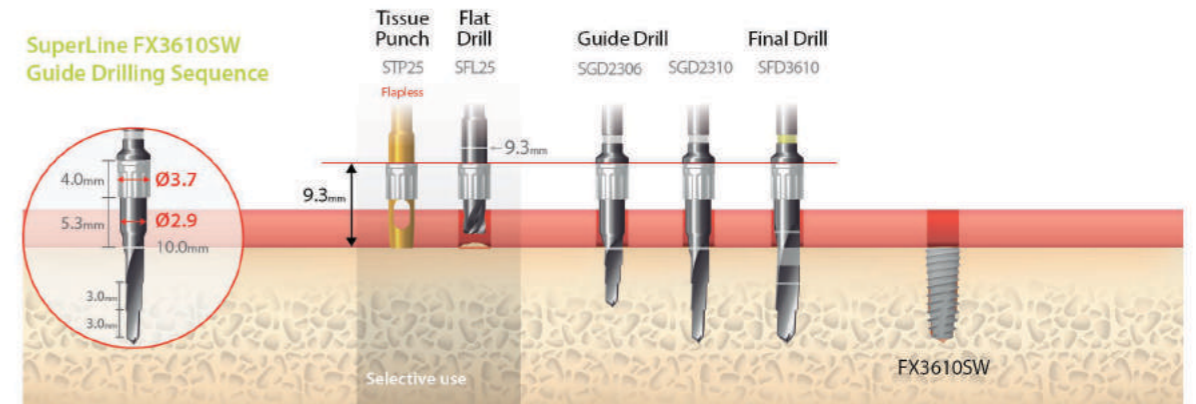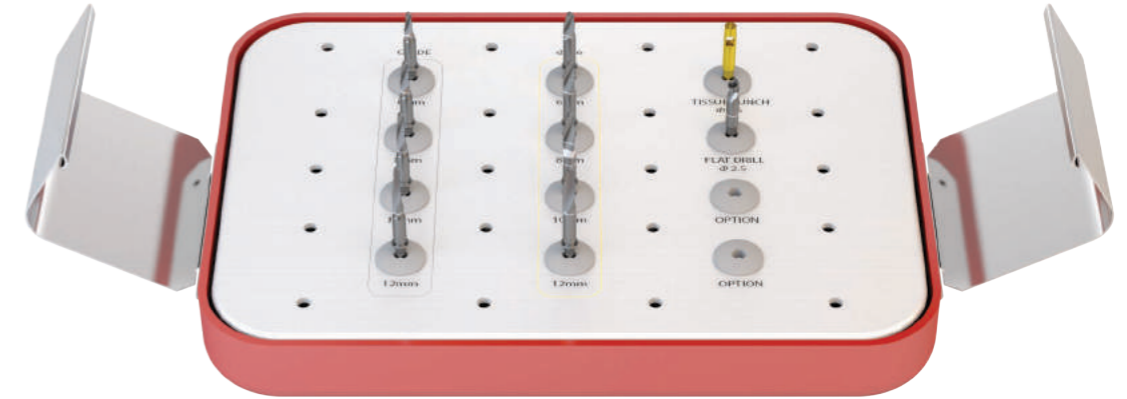





**Full Kit / Flapless Surgery** (SuperLine FX4510SW Installation)




**Tissue Punch**  
FTP45  
(1000rpm/30~45N-cm with irrigation)

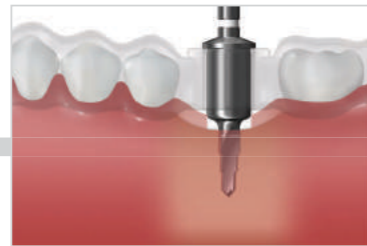


**Guide drill with Surgical guide**  
FGD 2306  
(1000rpm/30~45N-cm with irrigation)

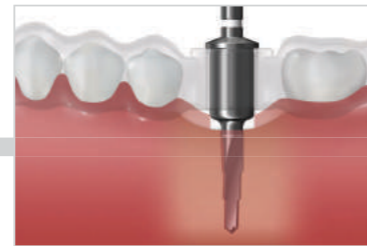


**Guide drill with Surgical guide**  
FGD 2310  
(1000rpm/30~45N-cm with irrigation)

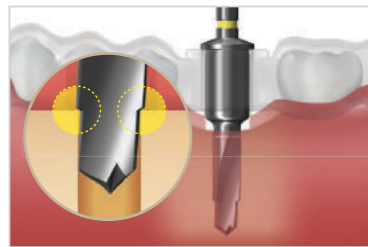


**Final drill with Surgical guide**  
FFD 3610  
(1000rpm/30~45N-cm with irrigation)

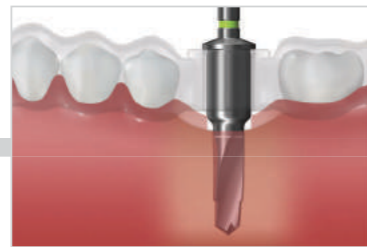

**Final drill with Surgical guide**  
FFD 4010  
(1000rpm/30~45N-cm with irrigation)

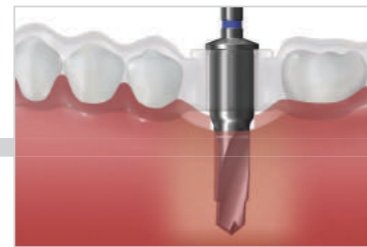


**Final drill with Surgical guide**  
FFD 4510  
(1000rpm/30~45N-cm with irrigation)

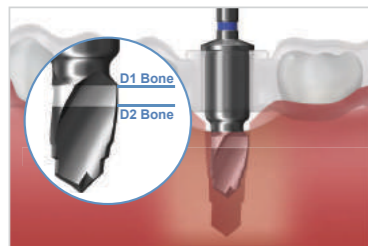


**Countersink with Surgical guide**  
FFD 4506  
(1000rpm/30~45N-cm with irrigation)




**Fixture installation with Surgical guide**  
FFA28H  
SuperLine : FX4510SW(50rpm/70N-cm)

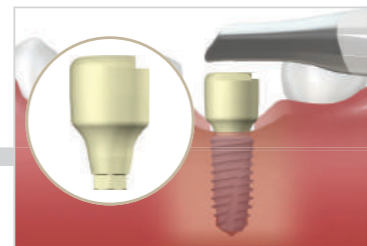


**IOS Healing Abutment**  
Connect IOS Healing Abutment & Take digital impression with intraoral scanner

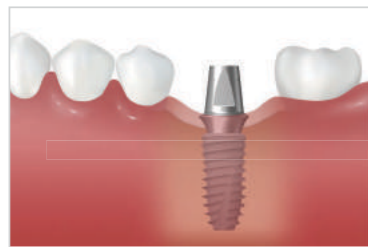

**Abutment selection**  
Select Stock abutment or Customized abutment regarding tissue condition and occlusion

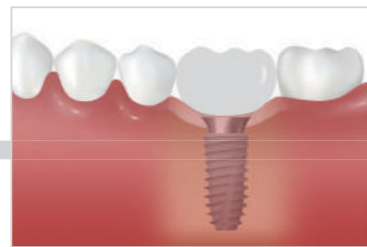


**Provisional restoration**  
Stabilize occlusion & Soft tissue

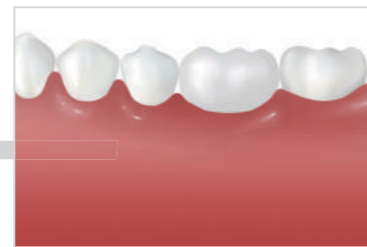


**Final prosthesis**


**Full Kit / Flap Surgery** (SuperLine FX5010SW Installation)

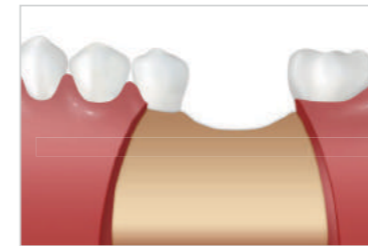


**Incision**

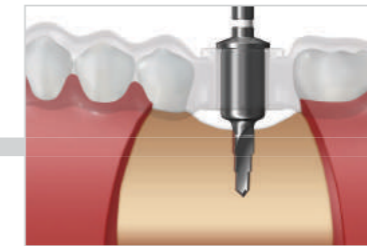


**Guide drill with Surgical guide**  
FGD 2306  
(1000rpm/30~45N-cm with irrigation)

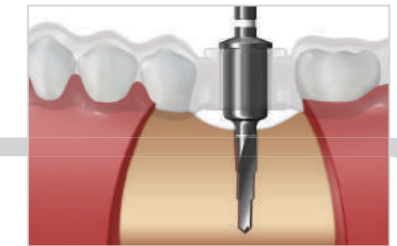


**Guide drill with Surgical guide**  
FGD 2310  
(1000rpm/30~45N-cm with irrigation)

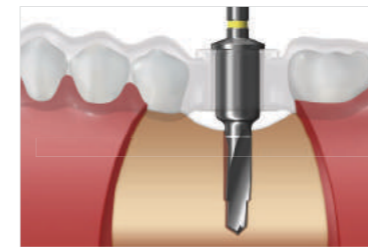


**Final drill with Surgical guide**  
FFD 3610  
(1000rpm/30~45N-cm with irrigation)

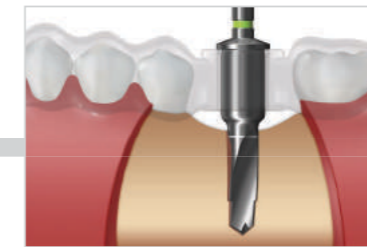



**Final drill with Surgical guide**  
FFD 4010  
(1000rpm/30~45N-cm with irrigation)

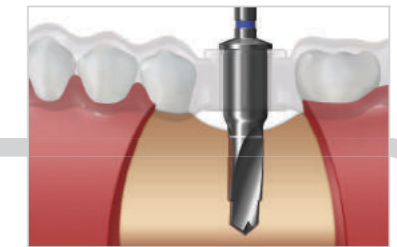


**Final drill with Surgical guide**  
FFD 4510  
(1000rpm/30~45N-cm with irrigation)

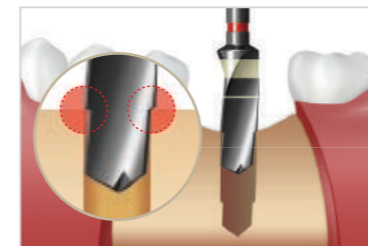


**SuperLine Final drill**  
XFD 5010  
(1000rpm/30~45N-cm with irrigation) or  
(50rpm/70N-cm without irrigation)

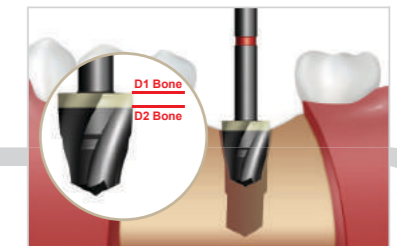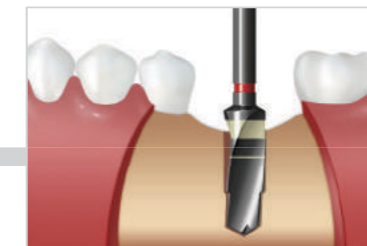


**SuperLine Countersink**  
XCS 5029SW  
(1000rpm/30~45N-cm without irrigation)

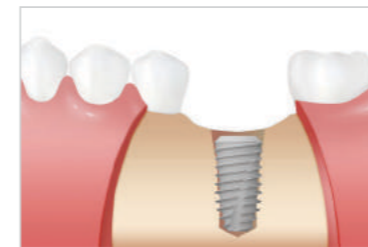


**Fixture installation**  
SuperLine : FX5010SW  
(50rpm/70N-cm)

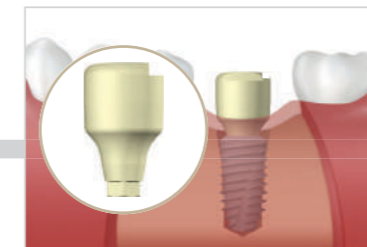


**IOS Healing Abutment**  
Connect IOS Healing Abutment & Take digital impression with intraoral scanner

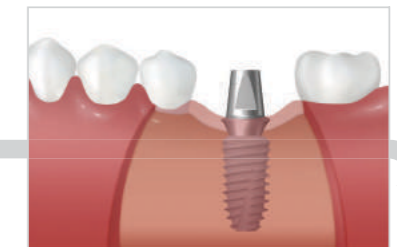

**Abutment selection**  
Select Stock abutment or Customized abutment regarding tissue condition and occlusion

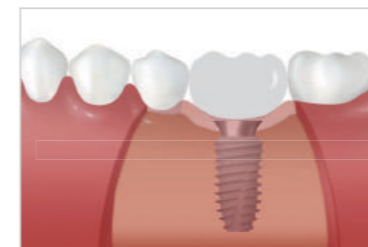


**Provisional restoration**  
Stabilize occlusion & Soft tissue

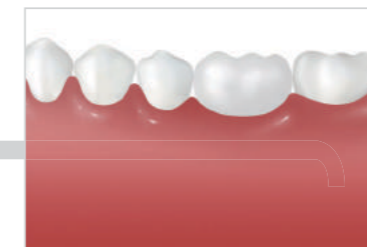


**Final prosthesis**


**Simple Kit / Flapless Surgery** (SuperLine FX3610SW Installation)

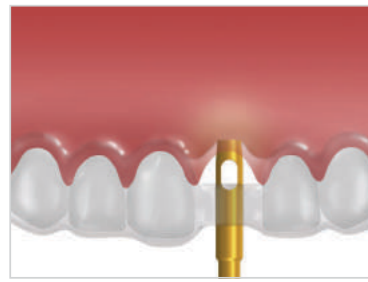

**Tissue Punch**  
STP25  
(1000rpm/30~45N-cm with irrigation)

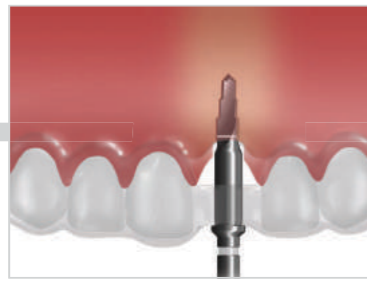


**Guide drill with Surgical guide**  
SGD 2306  
(1000rpm/30~45N-cm with irrigation)

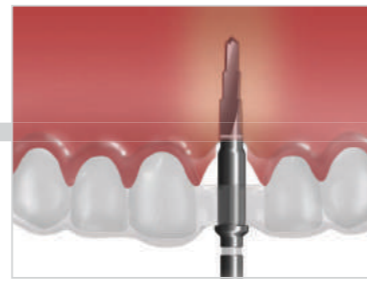


**Guide drill with Surgical guide**  
SGD 2310  
(1000rpm/30~45N-cm with irrigation)



**Final drill with Surgical guide**  
SFD 3610  
(1000rpm/30~45N-cm with irrigation)

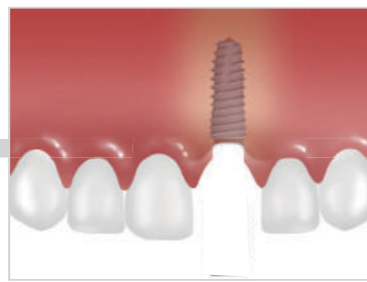

**Fixture installation**  
SuperLine : FX3610SW  
(50rpm/70N-cm)

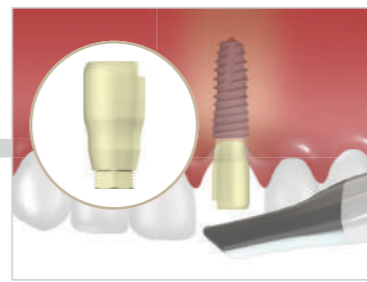


**IOS Healing Abutment**  
Connect IOS Healing Abutment & Take digital impression with intraoral scanner



**Abutment selection**  
Select Stock abutment or Customized abutment regarding tissue condition and occlusion




**Provisional restoration**  
Stabilize occlusion & Soft tissue

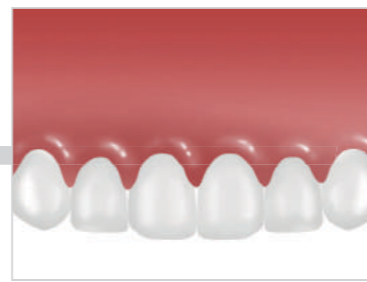


**Final prosthesis**


**Simple Kit / Flapless Surgery** (SuperLine FX4510SW Installation)

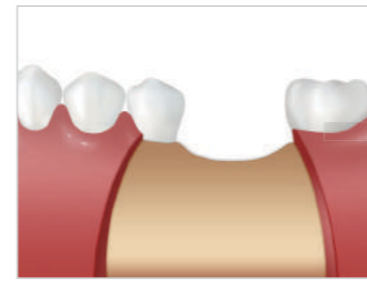


**Incision**

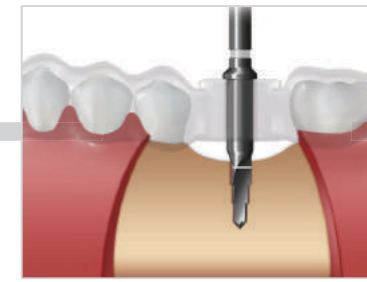


**Guide drill with Surgical guide**  
SGD 2306  
(1000rpm/30~45N-cm with irrigation)

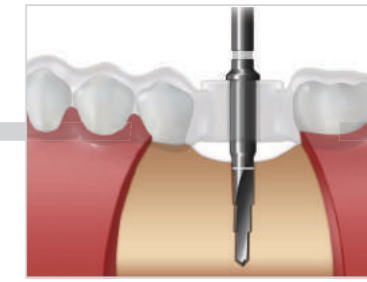



**Guide drill with Surgical guide**  
SGD 2310  
(1000rpm/30~45N-cm with irrigation)

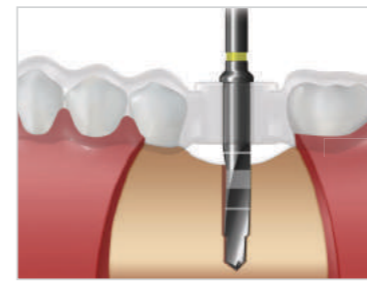


**Final drill with Surgical guide**  
SFD 3610  
(1000rpm/30~45N-cm with irrigation)

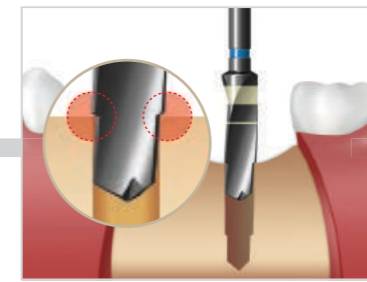


**SuperLine Final drill**  
XFD 4329  
(1000rpm/30~45N-cm with irrigation) or (50rpm/70N-cm without irrigation)

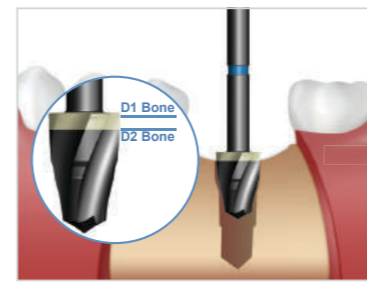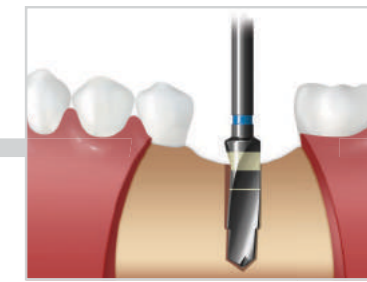


**SuperLine Countersink**  
XCS 4529SW  
(1000rpm/30~45N-cm without irrigation)

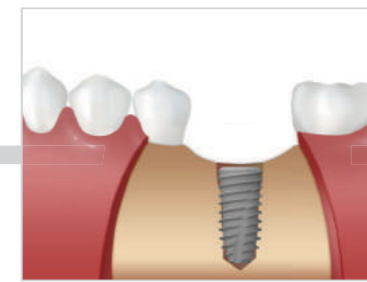


**Fixture installation**  
SuperLine : FX4510SW  
(50rpm/70N-cm)

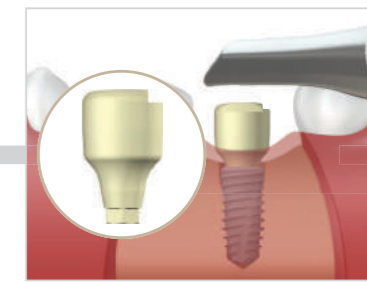


**IOS Healing Abutment**  
Connect IOS Healing Abutment & Take digital impression with intraoral scanner

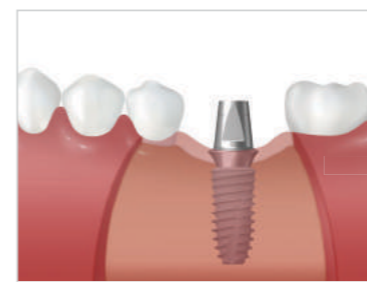

**Abutment selection**  
Select Stock abutment or Customized abutment regarding tissue condition and occlusion

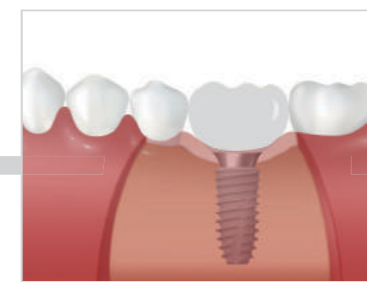

**Provisional restoration**  
stabilize occlusion & soft tissue

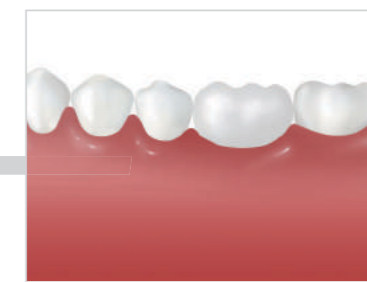

**Final prosthesis**



# Polymer Guide

## Thermoplastic Surgical Template for Dental Implant Placement

- Fabricate precise surgical template in just minutes using hot water
- Disposable material to promote control of infection and contamination
- Titanium sleeve is compatible with Dentium Final Drills





**Thermoplastic surgical template**  
Fabricate an accurate surgical template in just minutes, greatly reducing the lab and chairside time

**Fully compatible with Dentium drill system**

**Suitable template shape**  
Polymer Guide is designed to maximize contact area over the adjacent teeth while allowing substantial field of view during implant surgery



**Improved infection/contamination control**  
To ensure infection/contamination-free surgery every time, the Polymer Guide is made to be disposable

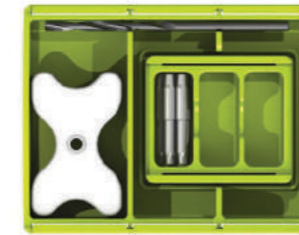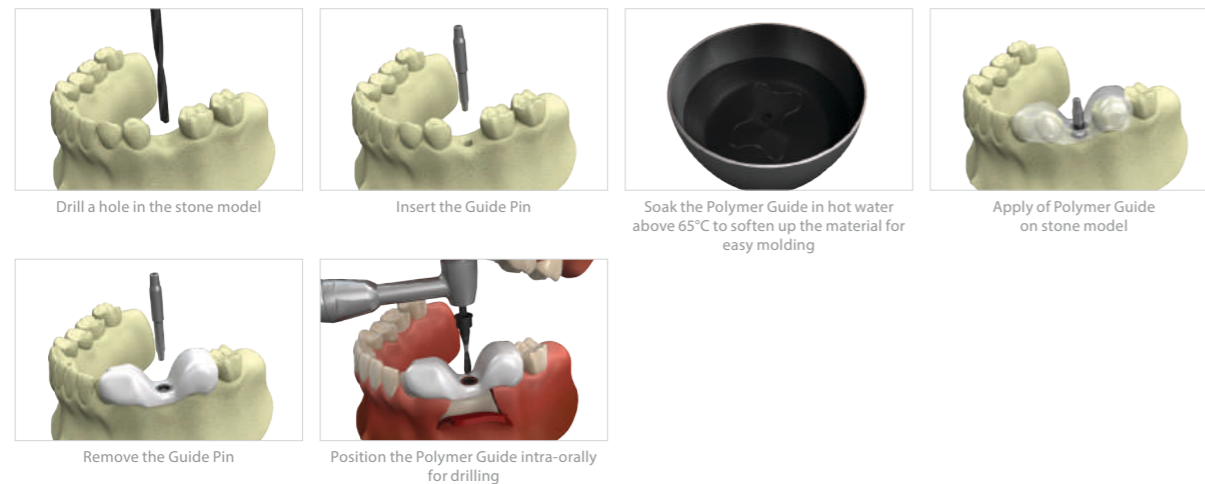

PGSSK



PGSCK

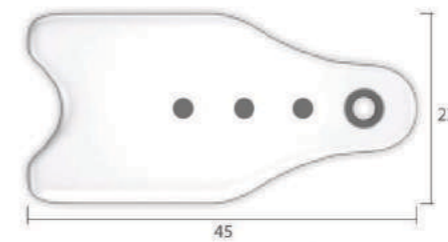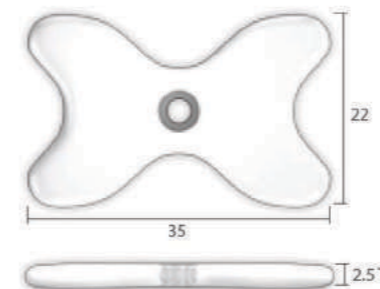

### Single Standard (5ea)

T	Art. No.
2.5	XSG 34 35 S

### Cantilever Multi-Ready (5ea)

T	Art. No.
2.5	XSG 34 45 C

\* 1 fixed Sleeve + 3 additional Sleeve Hole


- Stone Drill  XGD 23 60 (1ea)
- Guide Pin  XGP 34 23 S (5ea)
- Additional Metal Sleeve (for Cantilever Multi-Ready)  XPGS 34 25 A (5ea)



# Implant Guide Kit



Surgical guide implementing Silicon Spacer, Unique Single Guide pin & Multi Guide

- The Multi Guide is designed to take into consideration the mesiodistal width and drilling position in the edentulous area
- Single guide pins and Spacers are configured based on the average of tooth width

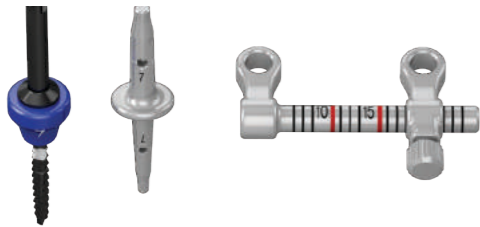

Assist in the implant positioning as well as determining the width and the position of the prosthetic components.

The Multi Guide is adjustable in length and angle to guide the implant position and drilling. It is particularly useful when applying multiple prostheses.

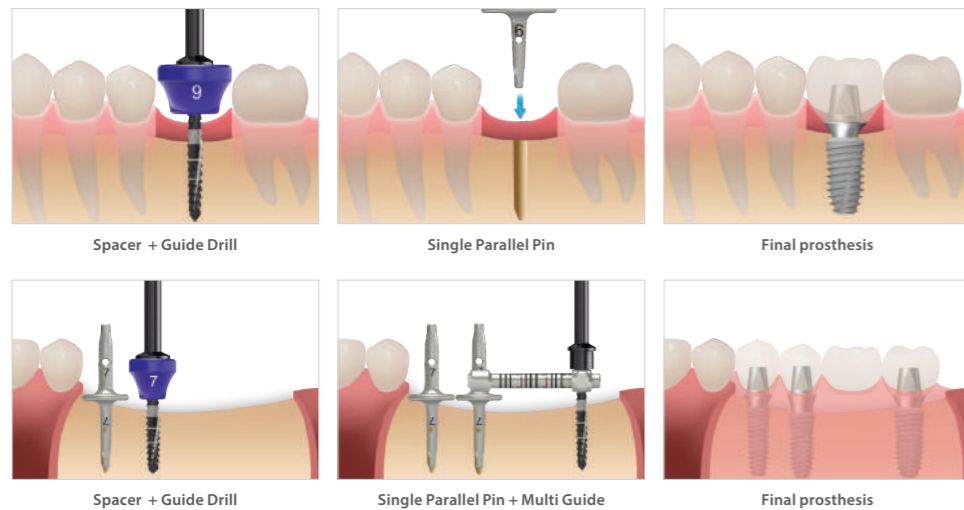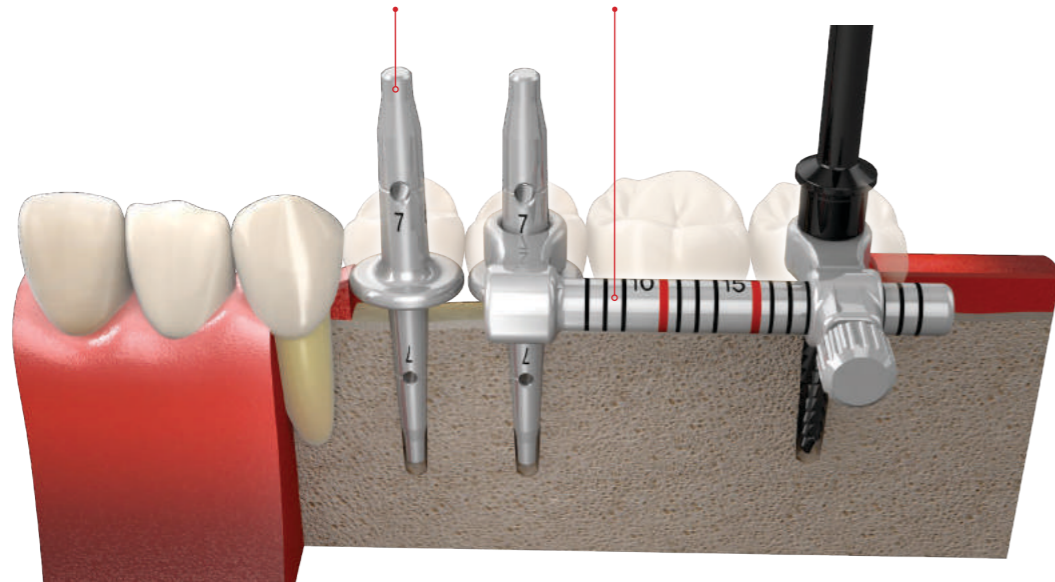

## GIGK

### Guide Drill

Diameter	L	Art. No.
Ø2.6	29	GGD 26 29
Ø2.6	35	GGD 26 35




### Spacer

Width	Art. No.
5.0	GSP 05
7.0	GSP 07
9.0	GSP 09
11.0	GSP 11



### Single Guide Pin

Width	Art. No.
4.0 / 5.0	GGP 04 05
6.0 / 7.0	GGP 06 07
8.0 / 9.0	GGP 08 09
10.0 / 11.0	GGP 10 11

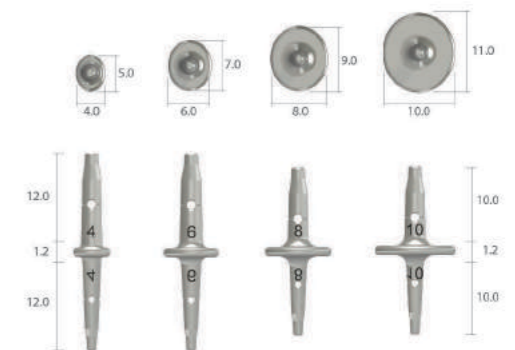


### Multi Guide


Art. No.	GMG2
----------	------



**Single Case** (SuperLine FX5010SW Installation)

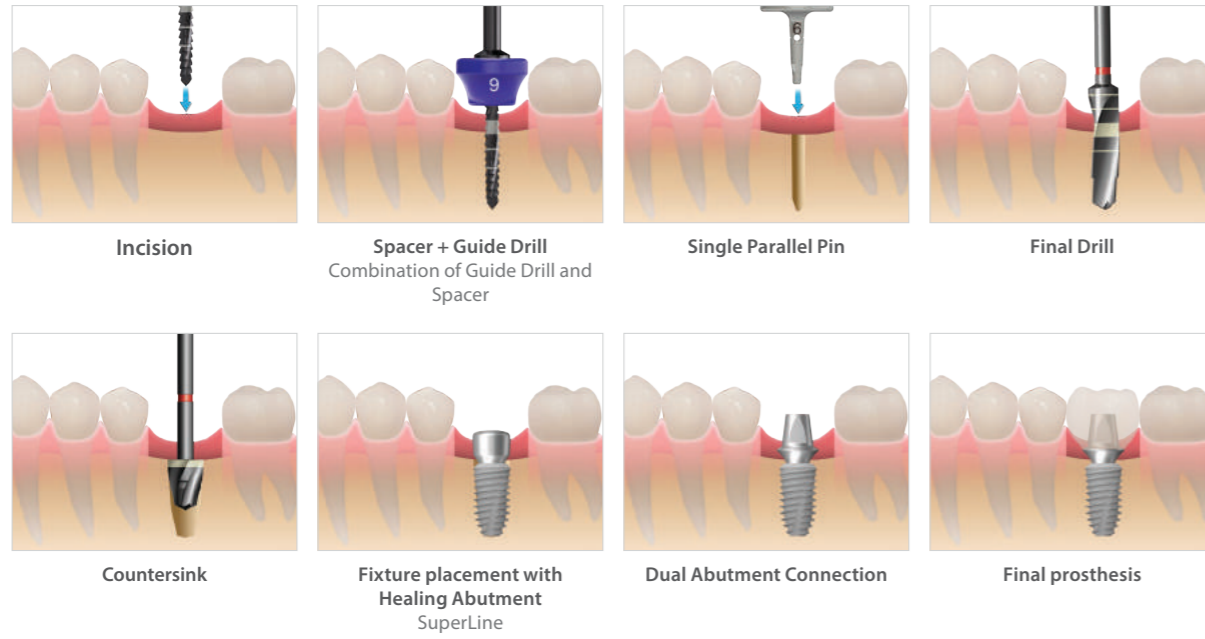

**Multiple Case** (SuperLine Installation)

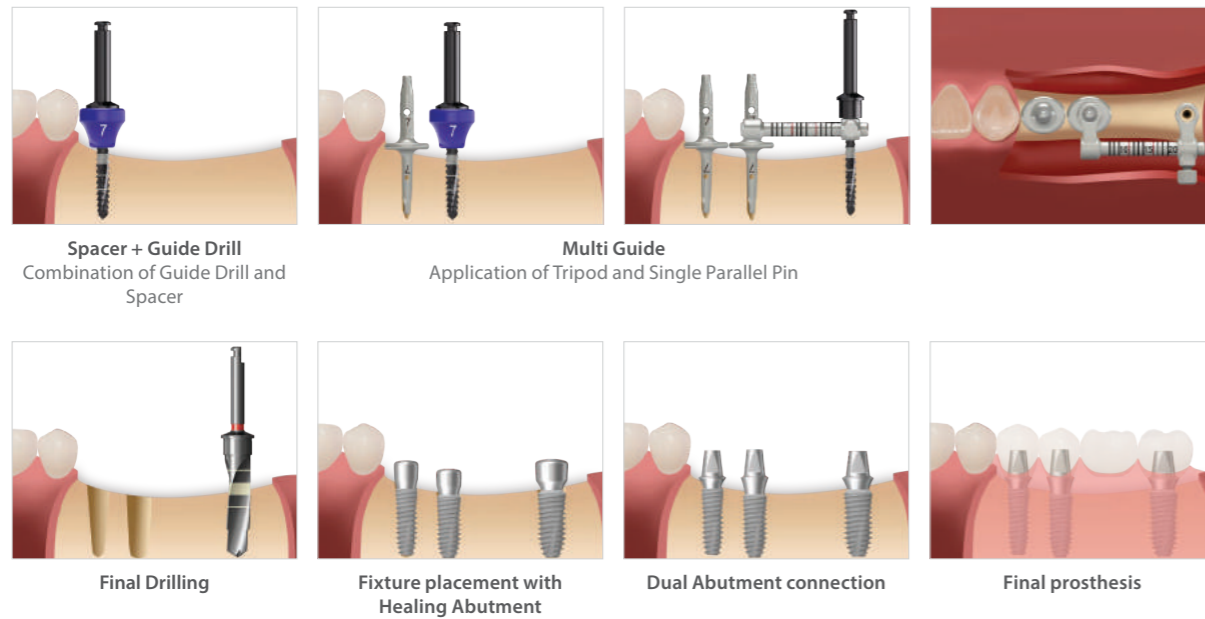


# Metal Kit

Sinus Kit  
RE Kit

# Sinus Kit

Unit: mm, Scale 1 : 1

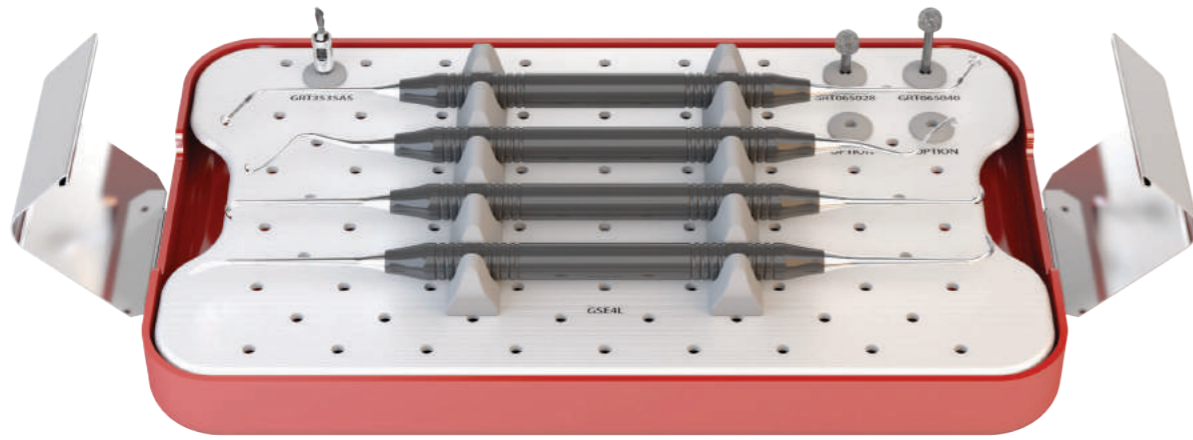

GSEK


## Crestal Drill

Diameter	L	Art. No.
Ø3.5	35	GRT 35 35 AS



## Lateral Drill

Diameter	L	Art. No.
Ø6.0	28	GRT 06 50 28
Ø6.0	40	GRT 06 50 40

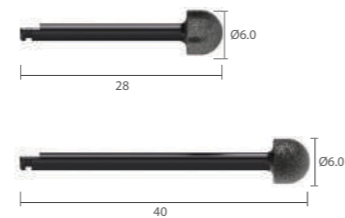


## Sinus Curette


Scale 1 : 2 / mm




## Crestal approach (SuperLine+ OSTEON™3 Lifting)

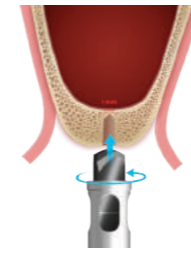

**Crestal Drill**  
After Final drilling, eliminate the residual bone [1mm] using a Crestal Drill at low speed [20~100rpm] until feel a slight drop

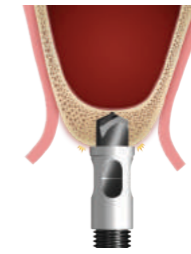


Adjust the desired length of drilling with Stopper to prevent unexpected Sinus perforation

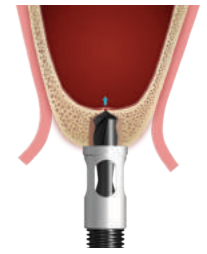


Eliminate the residual bone [1mm] using a Crestal Drill at low speed [20~100rpm] until feel a slight drop




Or eliminate the residual bone [1mm] using [Osteotome] until you feel a slight drop

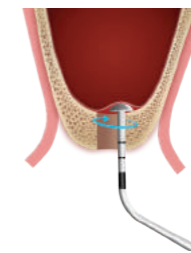


**Sinus Curette**  
Use the dome-shaped sinus curette (GSE 1L) to detach Sinus membrane

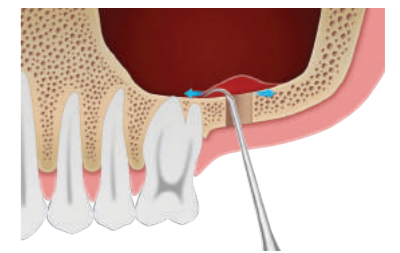


Elevate the sinus membrane to create adequate space for graft material

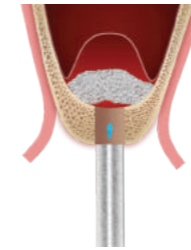


Fill the Sinus cavity with [OSTEON™3 Lifting] graft material

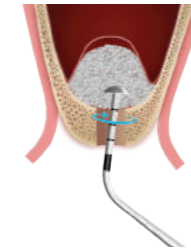

Fill and distribute [OSTEON™3 Lifting] graft material properly into the Created space

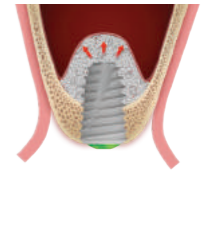


Fixture installation – [SuperLine] into the osteotomy site

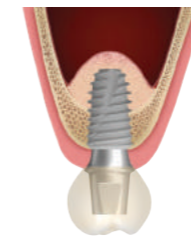

Final prosthesis




**Lateral approach** (SuperLine+OSTEON™3 Sinus+OSTEON™3 Collagen+Collagen Membrane)


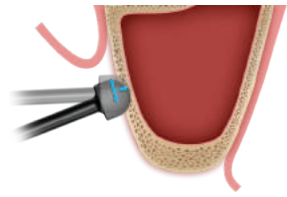


**Lateral Drill**

Thin down the lateral wall with Lateral Drill (Contra-Angle) to reach the Schneiderian membrane

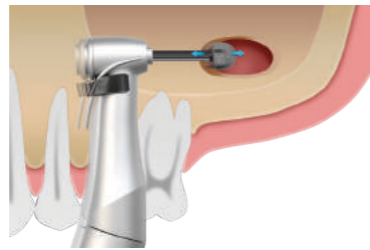


Mesio-distally with a gentle pressure until you get a proper size and shape of window for bone augmentation  
[Drill speed 800 to 1,200rpm, 30~45N-cm with irrigation]

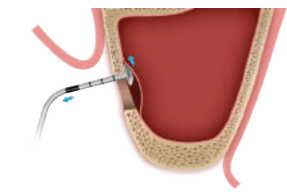
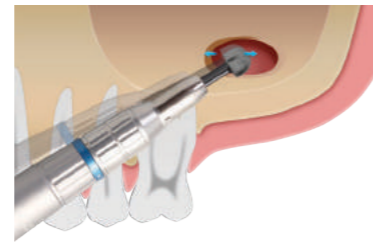


**Sinus Curette**


Detach sinus membrane using a dome-shape Sinus curette (GSE 1L)

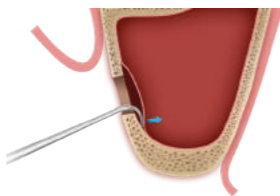


Elevate the sinus membrane to create adequate Space for graft material

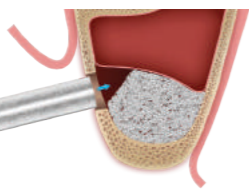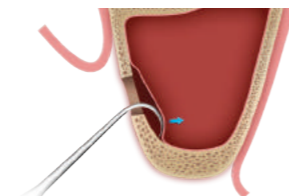


Fill the Sinus cavity with [OSTEON™3 Sinus] graft material

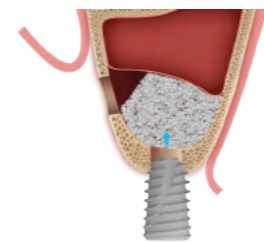

Fixture installation – [SuperLine] into the osteotomy site

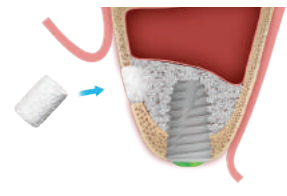

Fill into [OSTEON™3 Collagen] into lateral wall

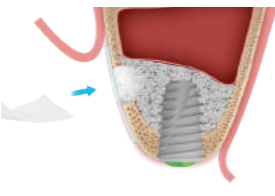


[Collagen Membrane] application



Final prosthesis

**RE (Ridge Expander) Kit**


Unit: mm, Scale 1 : 1

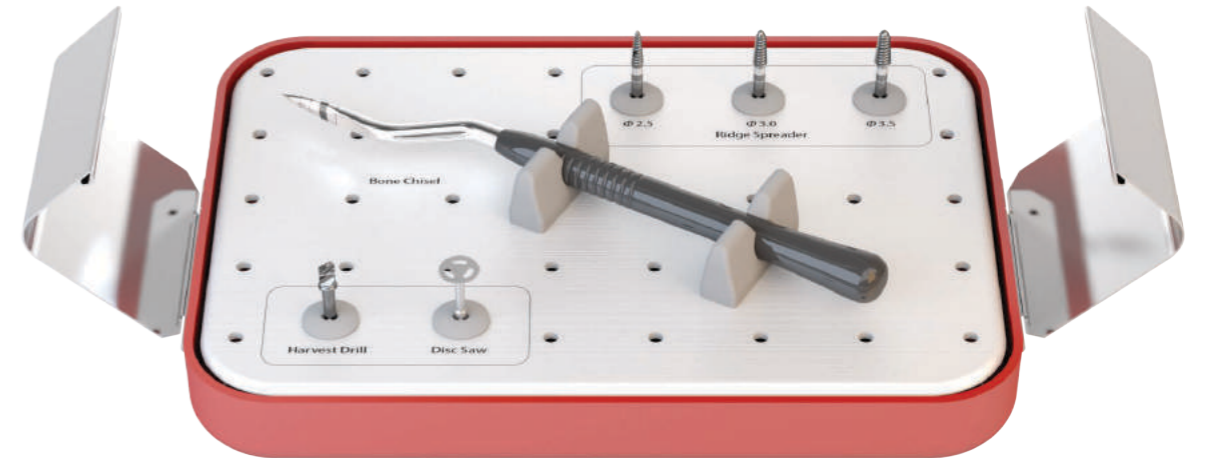

**GREK**


**Ridge Spreader**

Diameter	L	Art. No.
Ø1.3 / Ø2.5	32	GRS 13 25
Ø1.8 / Ø3.0	32	GRS 18 30
Ø2.3 / Ø3.5	32	GRS 23 35

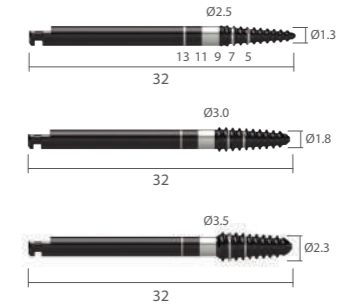


**Harvest Drill**

Diameter	L	Art. No.
Ø3.0	29	GHD 30 29

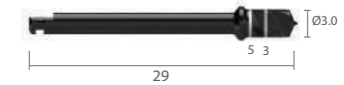

**Disc Saw**

Diameter	L	Art. No.
Ø8.0	25	GDS 80 25

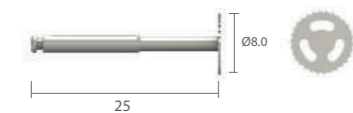


**Bone Chisel**

Art. No.	GBC 18 45 13
----------	--------------




# Dentium Advanced Sinus Kit (DASK)

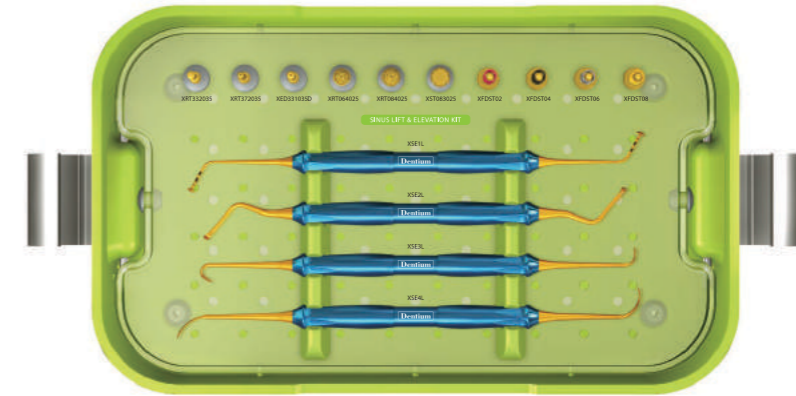

## Sinus Instruments

DASK  
Osteotome Kit  
Sinus Elevator

### DASK Drills

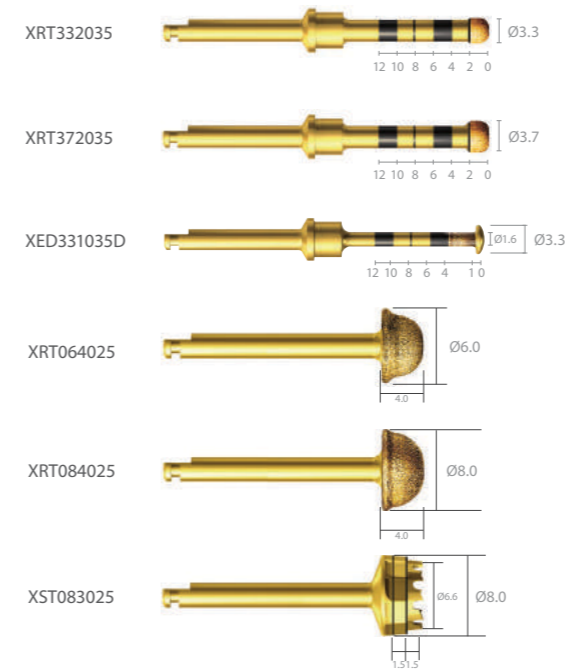

[Unit: mm, Scale 1.2 : 1]

Type	DASK Drill #	Art. No.
Crestal Approach	DASK Drill # 1	XRT <b>33</b> 2035
	DASK Drill # 2	XRT <b>37</b> 2035
	DASK Drill # 3	XED <b>33</b> 1035D
Lateral Approach	DASK Drill # 4	XRT <b>06</b> 4025
	DASK Drill # 5	XRT <b>08</b> 4025
	DASK Drill # 6	XRT <b>08</b> 3025

\* Note: Drill speed 800 to 1,200rpm, 30~45N-cm with irrigation


### Stoppers | for XRT332035, XRT372035, XED331035D



[Unit: mm, Scale 1 : 1]


Drilling Depth	L	Art. No.
<b>08</b>	10.6	XFDST <b>08</b>
<b>06</b>	12.6	XFDST <b>06</b>
<b>04</b>	14.6	XFDST <b>04</b>
<b>02</b>	16.6	XFDST <b>02</b>

### Sinus Bur Kit

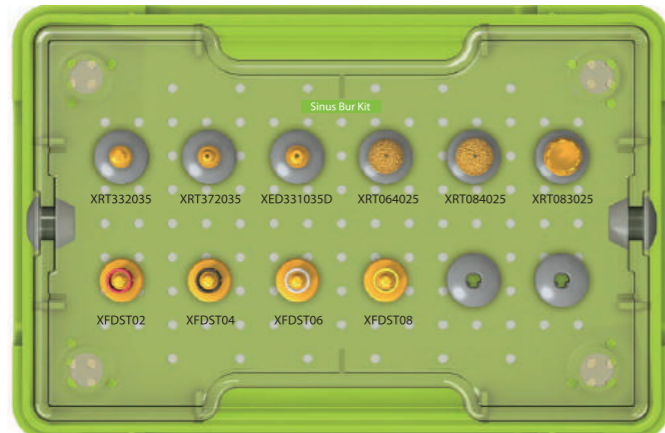

SDK


### Sinus Elevation Instruments

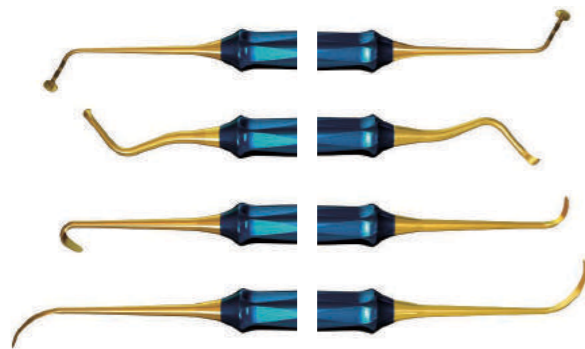

[Unit: mm, Scale 0.68 : 1 ]

Art. No.	XSE1L
Art. No.	XSE2L
Art. No.	XSE3L
Art. No.	XSE4L

### Drills for Crestal Approach

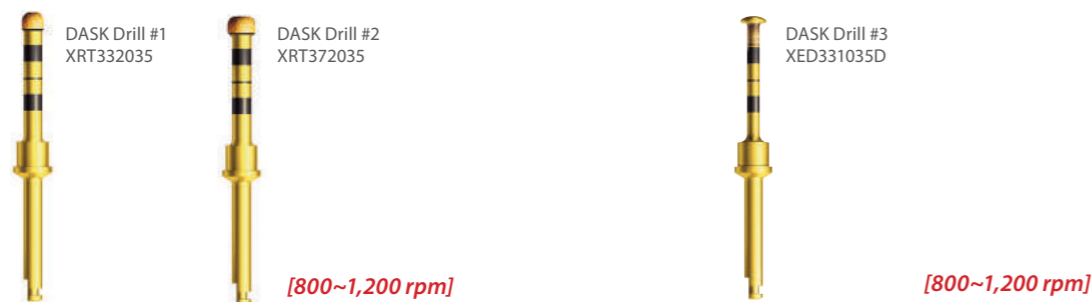


The distance from the alveolar crest to the sinus floor should be measured on x-rays prior to surgery. Site preparation is performed with twist drills in sequence up to 1mm short of the sinus floor. Then **DASK Drill #1** or **#2** is used and the sinus floor is carefully approached under light apical pressure. When you feel the sinus floor yields, remove the drill. Or partial preparations with **DASK Drill #1** or **#2** and up-fracture with osteotomes can be performed.

*\*The internal irrigation not only provides a cooling effect, but also adds hydraulic pressure to slightly lift the sinus membrane during drilling.*

When the sinus cavity is accessed, **DASK Drill #3** is introduced and much broader detachment from the sinus floor can be facilitated horizontally with hydraulic pressure thanks to the internal irrigation hole. **DASK Drill #3** can also be used for lateral window approach.

### Drills for Lateral Approach

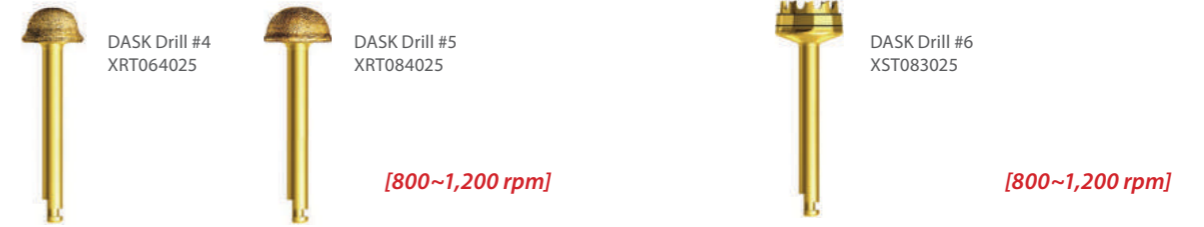

To make a lateral window through the anrostomy (thin-out) approach

*\*DASK Drill #4 or #5 is used to prepare a lateral sinus window using light pressure and rotating strokes. The DASK Drill #4 or #5 is designed to minimize the risk of sinus membrane perforation.*

To make a lateral window through the wall-off technique

*\*DASK Drill #6 is used to cut and detach a bony island like a trephine bur from the lateral wall. Uncontrolled overdrilling may lead to sinus perforation and possible damage to the membrane. External irrigation is necessary when drilling.*


### DASK Maintenance

#### Sterilization and Instrument Care Procedures


- It is important to use protective clothing and face shield while cleaning contaminated instruments.
  - Always wear protective glasses, mask, gloves, etc. for your safety.
1. Rinse instruments immediately after use under running tap water (<40° C) for a minimum of one (1) minute to remove all debris including extraneous body uids, bone debris and tissue.
  2. Soak all instruments immediately after rinsing in an enzymatic cleaning solution \* for 10 to 20 minutes (Do not soak overnight.)  
\*Follow manufacturer's instructions and observe recommended cleaning solution concentrations (enzymatic detergent with a pH level between 7-10 and temperature not to exceed 40° C). Do not use incompatible cleaning solutions to clean instruments.
  3. For internal irrigation drills, use a 1mL syringe and a 25 gauge needle to clean the drill irrigation hole with a minimum of 0.2mL of the prepared cleaning solution. Repeat this step two (2) more times for a total of three (3) rinses.
  4. of the prepared cleaning solution. Repeat this step two (2) more times for a total of three (3) rinses.
  5. Scrub with a soft brush a minimum of 1 (one) minute to remove any debris from the drill.
  6. Rinse instruments under running tap water 9<40°C) for a minimum of 1 minute. Use a 1mL syringe and a 25 gauge needle with a minimum of 0.2mL of tap water to forcefully ush inside the drill irrigation hole. Repeat ushing of drill irrigation hole two(2) more times for a total of three (3) ushes.
  7. Place instruments into an ultrasonic cleaner with neutral detergent\*\*. Keep instruments inside the ultrasonic bath for 15 minutes using a frequency of 25-50 kHz. Ensure multiple instruments placed within the bath remain separated.  
\*\*Follow manufacturer's instructions and observe recommended neutral detergent solution concentrations (neutral detergent with a pH level between 7-10 and temperature not to exceed 40°C). Do not use incompatible neutral detergent solutions to clean instruments.
  8. Rinse instruments thoroughly with running tap water(<40°C)for a minimum of 1 (one) minute until all traces of neutral detergent solution are removed. Rinse inside drill irrigation hole using a 1mL syringe and a 25 gauge needle with a minimum of 0.2mL of tap water. Repeat rinsing drill irrigation hole two (2) more times for a total of three (3) rinses.
  9. Gently wipe instruments with a soft lint-free cloth or place the instruments in a drying cabinet (60°C for less than 10 hours) until fully dry. Blow residual water from drill irrigation hole using a 1mL syringe and a 25 gauge needle. Visually inspect instruments in a well-lit area to ensure they are clean, dry and free of residue.
  10. a well-lit area to ensure they are clean, dry and free of residue.
  11. Clean instrument trays with a germicidal cleaner prior to returning instruments into Kit.
  12. Always check for damage or corrosion after rinsing and dryin




**Crestal Approach (Sinus Lifting)** (SuperLine+OSTEON™3 Lifting)

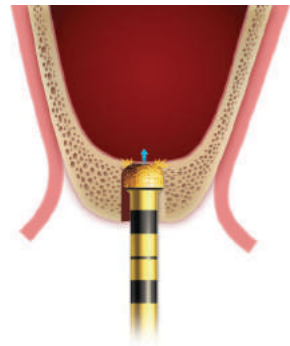


**DASK Drill**  
After Ø3.8 Final drilling, eliminate the residual bone (1mm) using a DASK Drill #1 or #2 (in hard bone) until you feel a slight drop

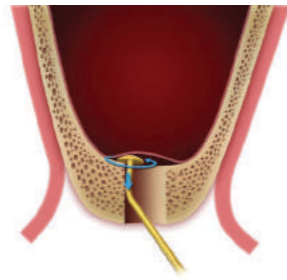


**Sinus Elevation Instruments**  
Detach the sinus membrane to create adequate space for graft material using dome-shaped curette

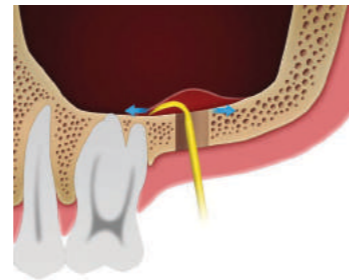


Elevate the Sinus membrane to create adequate space for graft material

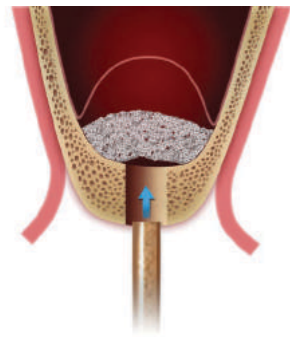


Fill the sinus cavity with [OSTEON™3 Lifting] graft material

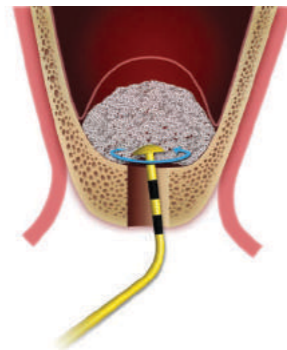

Fill and distribute [OSTEON™3 Lifting] graft material properly into the Created space

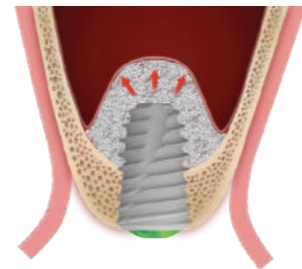


Fixture installation – [SuperLine] into the osteotomy site

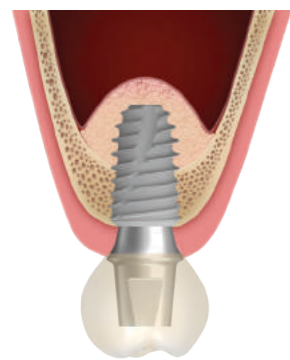


Final prosthesis

**Lateral Approach (Sinus Elevation)** (SuperLine+OSTEON™3 Sinus)

**Wall-off Technique**

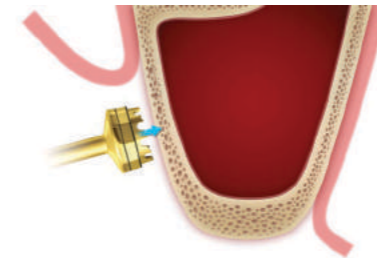


DASK Drill#6 is used to cut a round bony island from the lateral wall like a trephine bur.  
Start to drill at a desired location and proceed until you see the shadow of the sinus membrane. Then, separate and lift the bony island up from the neighboring wall with a molt curette or a periosteal elevator.  
The bony island is repositioned back in its original after bone augmentation

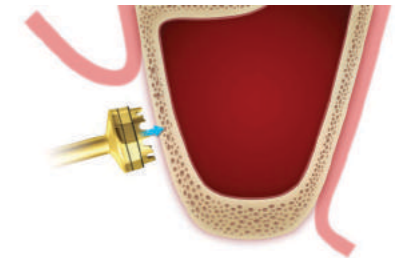


The first laser mark is 1.5mm and the second is 3.0mm.  
Overdrilling can cause sinus perforation and possible damage to the membrane



**Thin-out Technique**

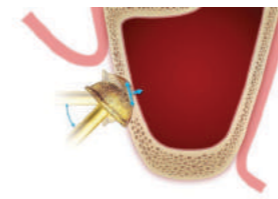


Thin down the lateral wall with DASK Drill #4 or #5 at a 45 degree angle to reach the Schneiderian membrane

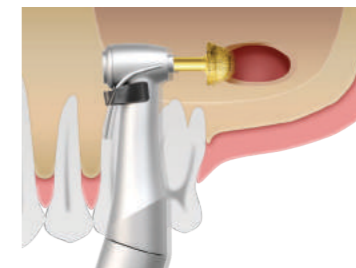



Move the DASK Drill #4 or #5 Mesio-distally with a gentle pressure until You get a proper size and shape of the window for bone augmentation

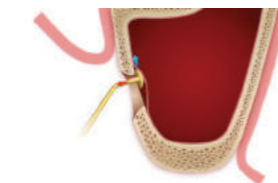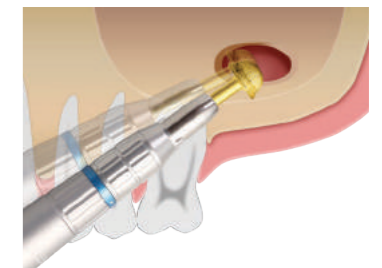


Sinus Elevation Instruments Detach sinus membrane using a dome-shape Sinus curette

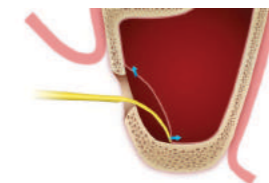


Elevate the sinus membrane to create adequate space for graft material

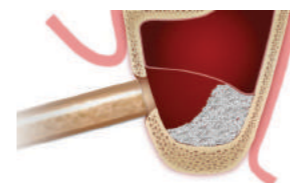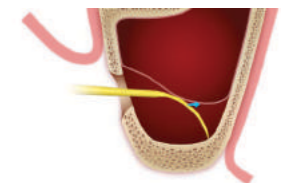

Fill the Sinus cavity with [OSTEON™3 Sinus] graft material

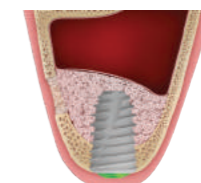

Fixture installation – [SuperLine] into the osteotomy site

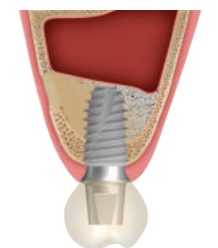


Final prosthesis

# Osteotome Kit

- Osteotomes compress the bone laterally, providing denser bony interface rather than removing valuable bone from the surgical site

## Osteotomes | Final drill type

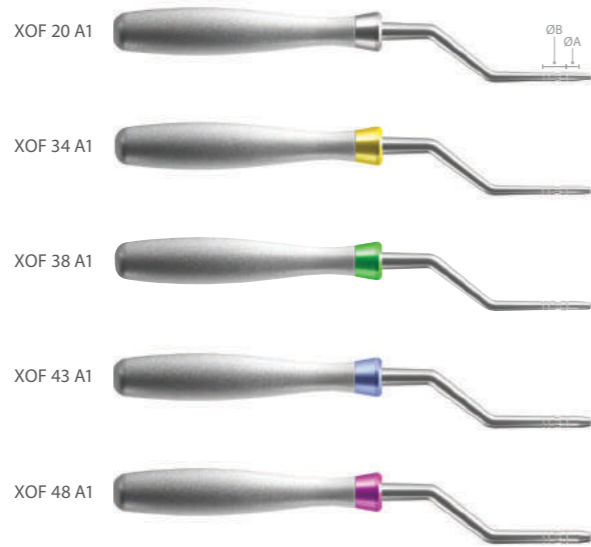

[ Unit: mm, Scale 0.4 : 1 ]

Type A  
(Convex)



Type B  
(Concave)






Type	Art. No.	ØA	ØB
XOFK <b>A</b> (Convex)	XOFK 20 A1	Ø1.7	Ø2.8
	XOFK 34 A1	Ø2.3	Ø2.8
	XOFK 38 A1	Ø2.7	Ø3.2
	XOFK 43 A1	Ø2.8	Ø3.8
	XOFK 48 A1	Ø3.0	Ø4.3
XOFBK <b>B</b> (Concave)	XOFK 20 B1	Ø1.7	Ø2.8
	XOFK 34 B1	Ø2.3	Ø2.8
	XOFK 38 B1	Ø2.7	Ø3.2
	XOFK 43 B1	Ø2.8	Ø3.8
	XOFK 48 B1	Ø3.0	Ø4.3

# Sinus Elevator

- Makes the sinus lift easy and drastically reduce the possibility of membrane perforation
- Balloon expansion of 0.5cc saline equals 6mm of membrane elevation




Type	Art. No.
Including Syringe	GSB 38
Balloon Only	GSB 38B

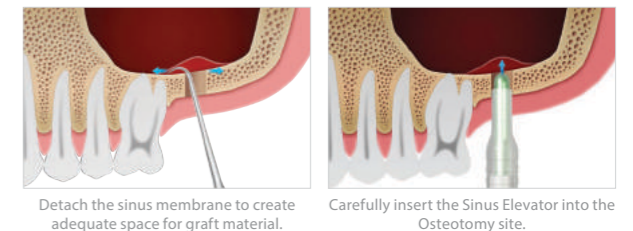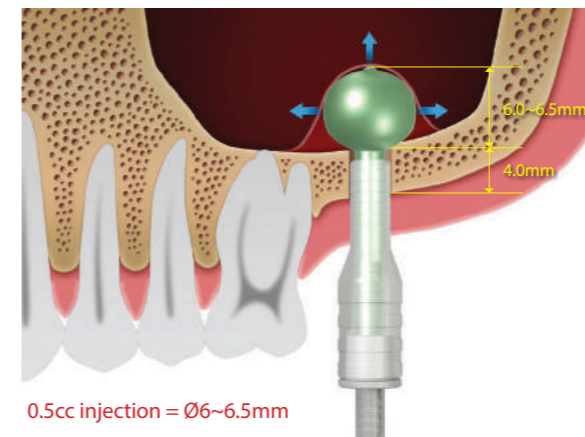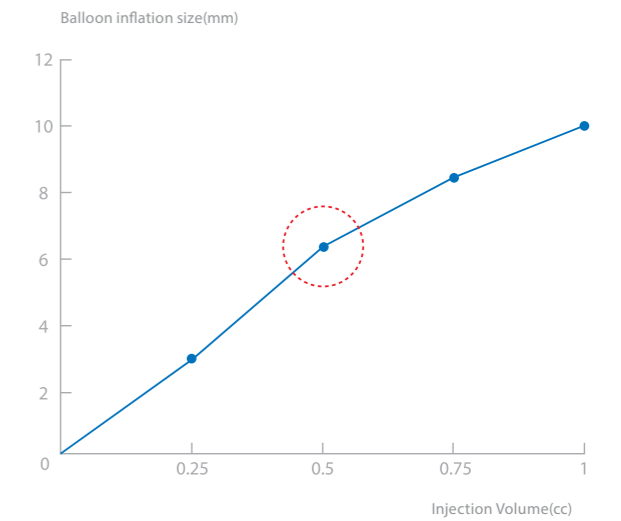

## RS (Ridge Spreader) Kit

- Allows the achievement of space for implantation through the spreading of the bone with chisel without drilling
- There are three types of Ridge Spreaders to create space up to Ø4.5mm
- Convenient surgeries due to the compatibility with hand-piece and ratchet
- Easy-to-use kit component

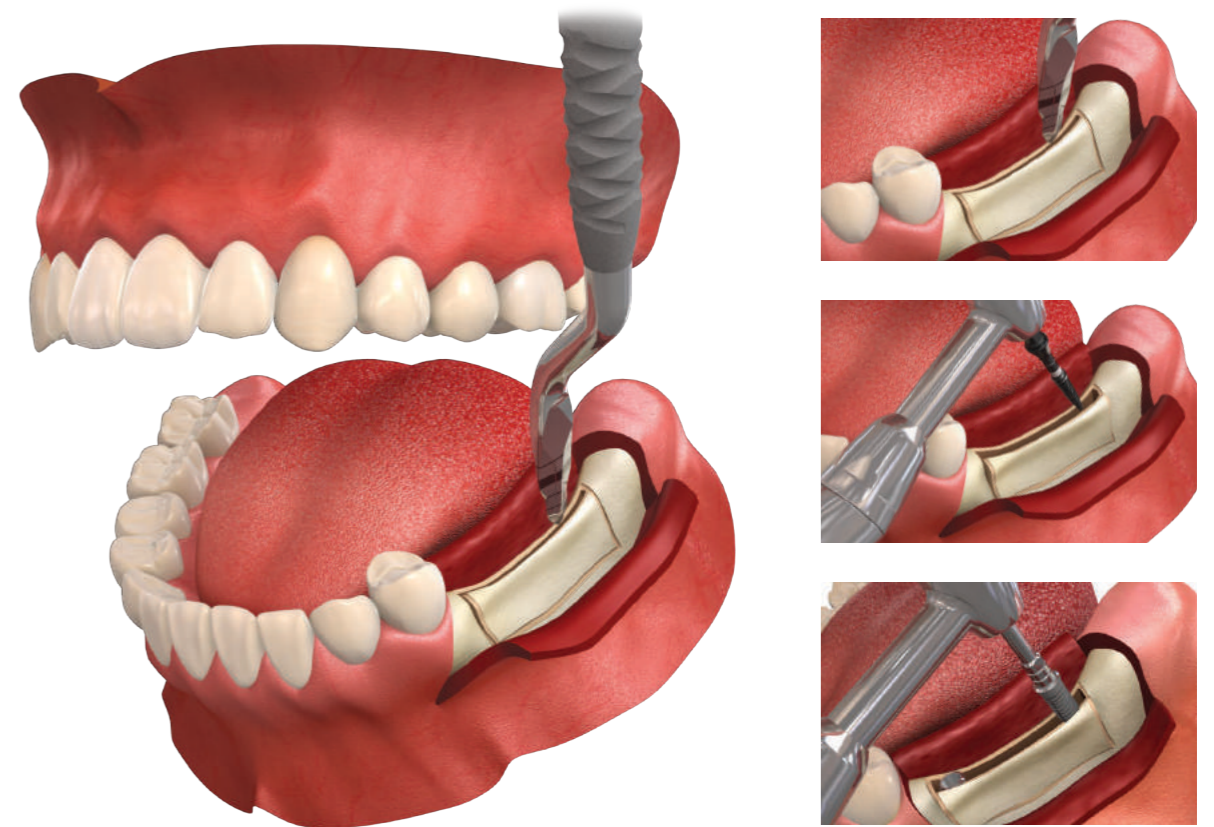


## GBR Instruments

RS Kit  
Harvest Drill






# RS (Ridge Spreader) Kit

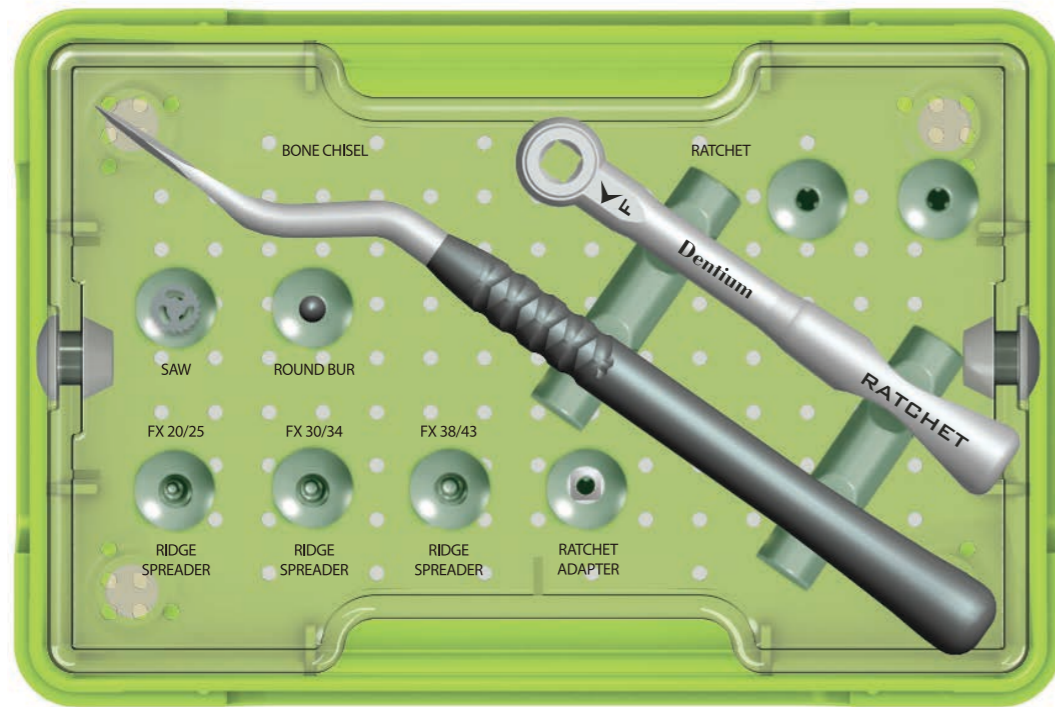

XRSK

**Kit contents**

Bone Chisel		XBC305013
Ratchet		XRCA1
Ridge Spreader Drills		RS142435
		RS203235
		RS263635
Round Bur		XRB4035
Ratchet Adapter		XRA3917
Mini Saw		XDS8025

## Bone Chisel



Art. No.	XBC305013
----------	-----------


## Ratchet



Art. No.	XRCA1
----------	-------

[Unit: mm, Scale 0.6 : 1]

## Ridge Spreader Drills

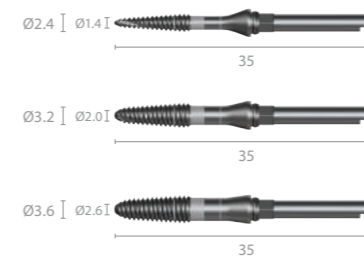

Diameter	L	Art. No.
Ø1.4 / Ø2.4	35	RS142435
Ø2.0 / Ø3.2	35	RS203235
Ø2.6 / Ø3.6	35	RS263635

## Round Bur




Diameter	L	Art. No.
Ø4.0	35	XRB4035

## Ratchet Adapter



Art. No.	XRA3917
----------	---------


## Mini Saw

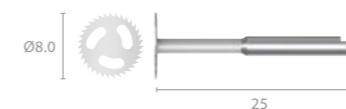

Diameter	L	Art. No.
Ø8.0	25	XDS8025

[Unit: mm, Scale 1 : 1]


**Ridge Spreading** (NR Line + OSTEON™ 3 + Collagen Membrane)

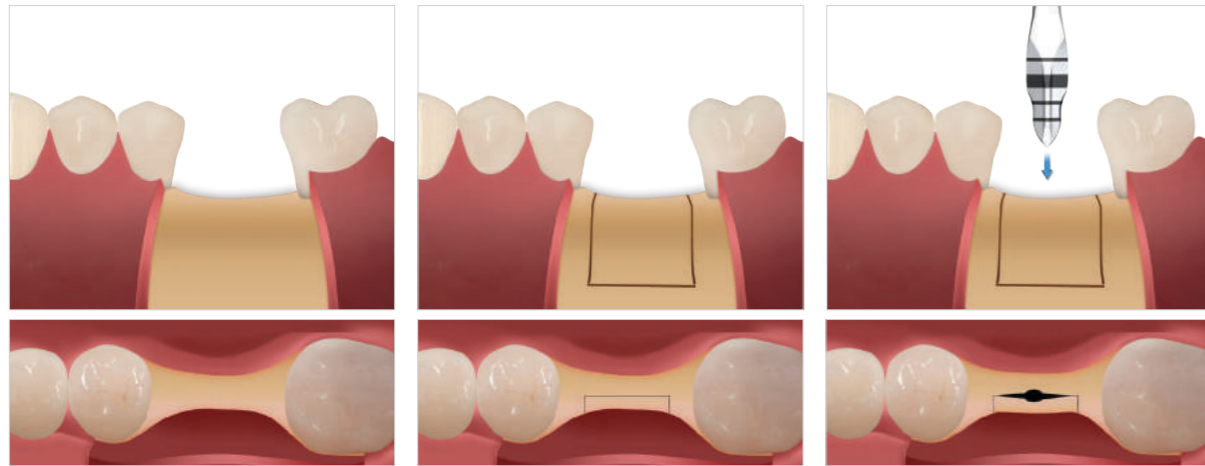


Incision

Decortification

Expansion with Bone Chisel

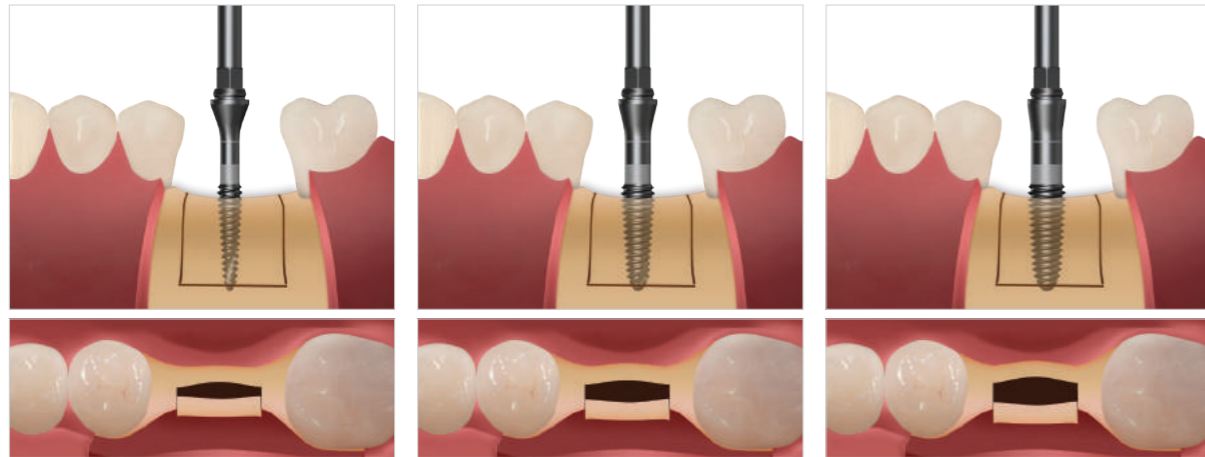


Expansion with Ridge Spreader (20~60rpm / 30~45N·cm) - Expanding alveolar bone ridge to make space for fixture

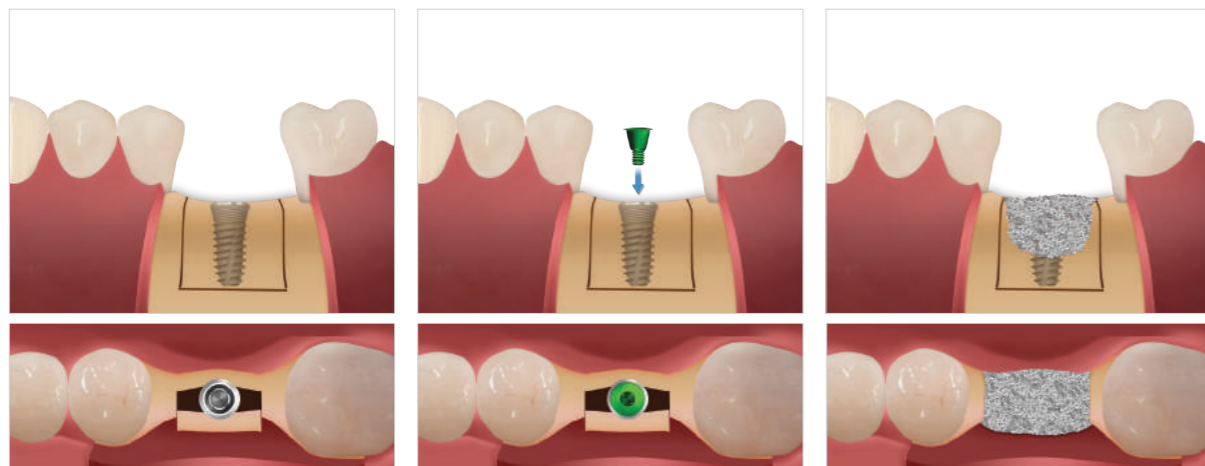

Fixture placement  
NR Line

Cover Screw connection


Application of graft material  
OSTEON™ 3

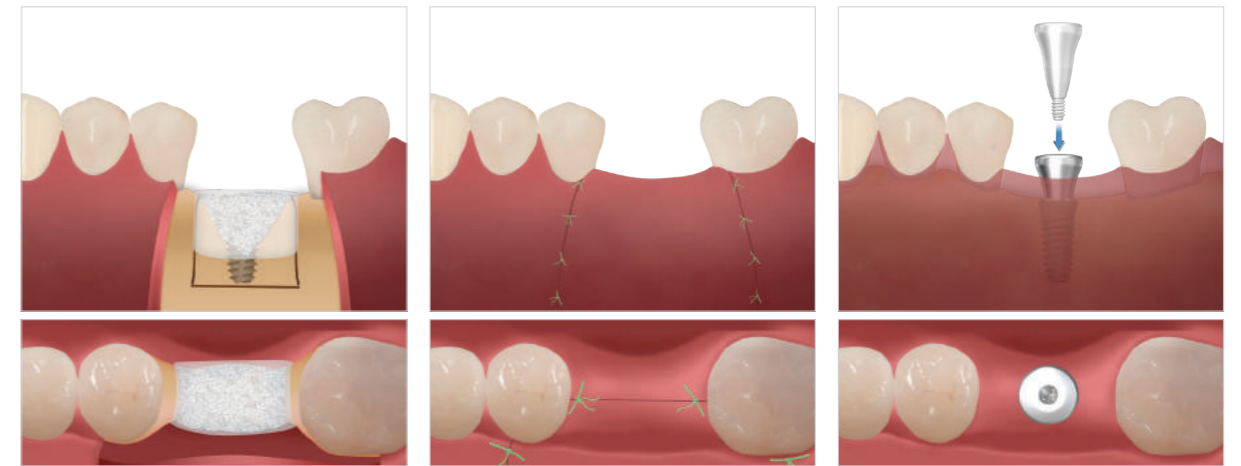

Barrier membrane application  
Collagen Membrane

Suture

Healing Abutment connection

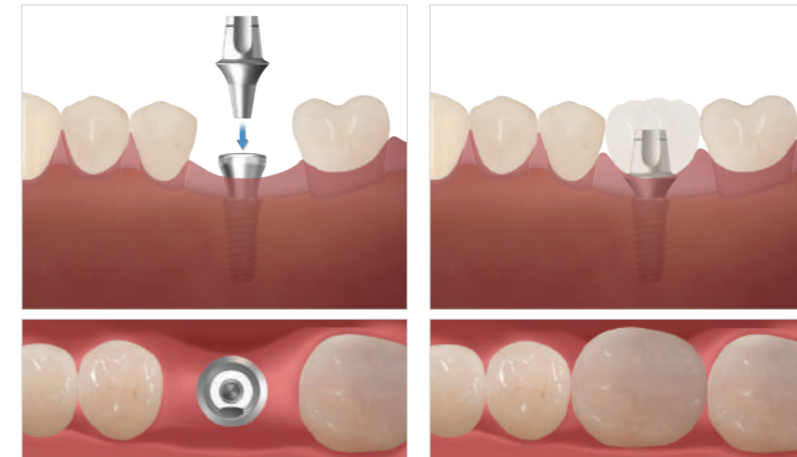

Dual Abutment connection


Final prosthesis

# Harvest Drill

Collect autogenous bone and prep osteotomy simultaneously and effectively using the specially designed drills, the Harvest Drills


- Sharp, pointed tip to prevent drill chattering for precise drilling
- Drill stoppers applicable to control the depth of the drilling for safe and efficient bone harvesting, especially in the buccal side of the ridge
- Recommended drill speed of less than 100 rpm / 50N-cm helps preserve the vital autogenous bone
- Excellent clinical results may be achieved when harvested autogenous bone is combined with Osteon™ II

## Harvest Drills

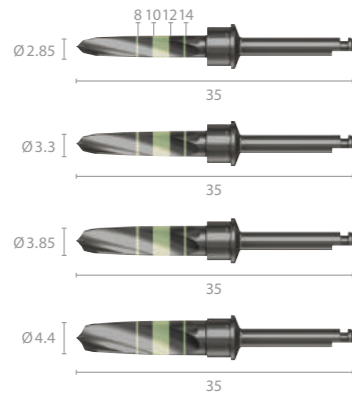


Diameter	L	Art. No.
Ø2.85	35	XFH 34 35
Ø3.3	35	XFH 38 35
Ø3.85	35	XFH 43 35
Ø4.4	35	XFH 48 35

## Harvest Drill Stopper

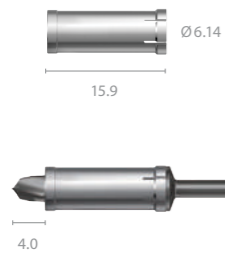


Diameter	L	Art. No.
Ø6.14	15.9	XFHST04

## Final Drill

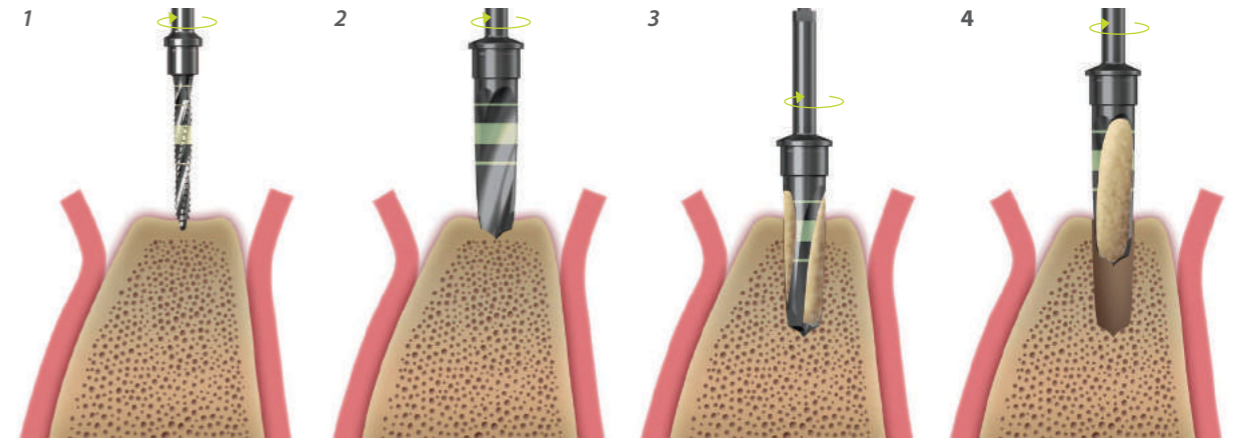

1 - First Guide Drill  
1,000rpm / 30~45N-cm with irrigation

2 ~ 4 - Harvest Drill  
30~100rpm / 30~50N-cm without irrigation

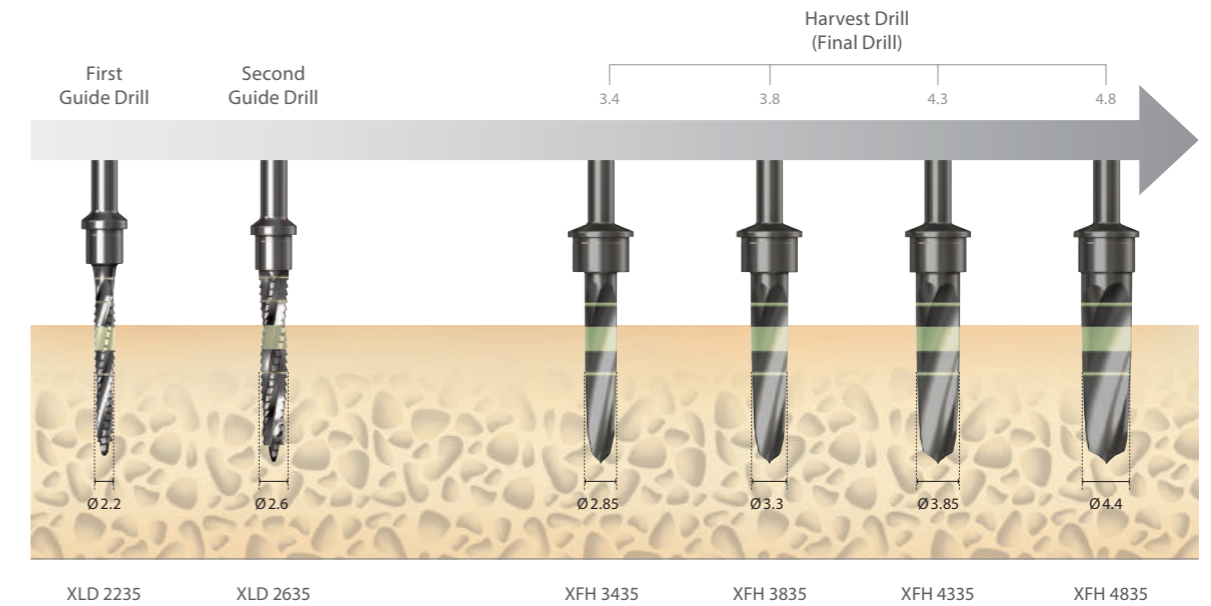


\* During the 4.3/4.8 fixture insertion into the bone density of D3~D4, the 3.35/3.85 harvest drilling process can be skipped.



# Help Kit

- Easy solution for critical problems which may occur in the prosthetic process consist of 5 tools in a kit
- (Screw Remover/ Abutment Hex Remover/ Screw Tap Repair / Fixture Remover / Cover & Abutment Screw Remover)
- Compatible with most dental implant products now available on the global market
- Heavy duty with robust design and proven materials

**IMPLANTIUM**  
**SuperLine**  
**SimpleLine II**

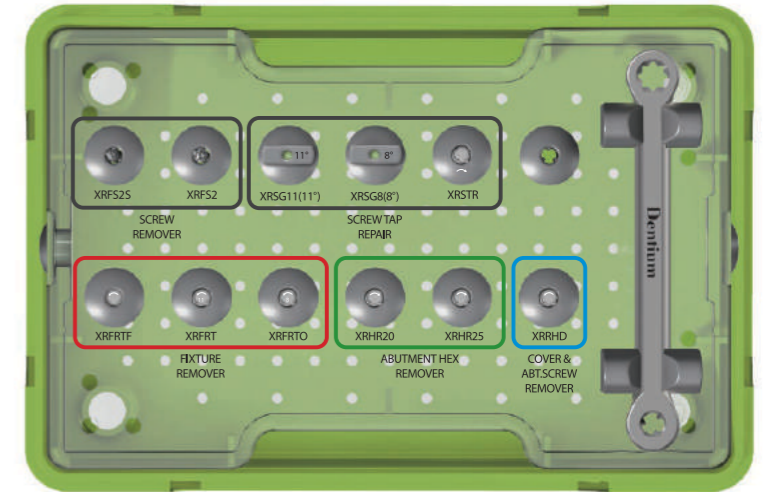

XIH

## Others

*Help Kit  
 Temporary Shell  
 White Seal  
 TN-Brush*


### Screw Remover

L	Art. No.
25	XRF S2S
35	XRF S2



### Abutment Remover

L	Art. No.
20	XRHR 20
25	XRHR 25

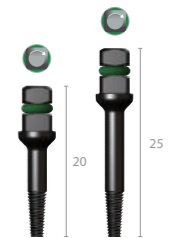


### Screw Tap Repair

Type	Art. No.
Tap	XRSTR
11° Guide	XRS G11
8° Guide	XRS G8




### Fixture Remover

Type	Art. No.
	XRFRTF
Remover	XRFRT
	XRFRT0
Wrench	XRFRW

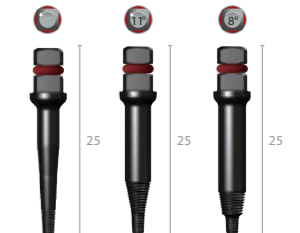


### Cover & Abutment Screw Remover

L	Art. No.
25	XRRHD

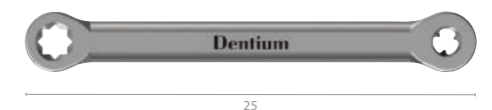

# NR Lrñe

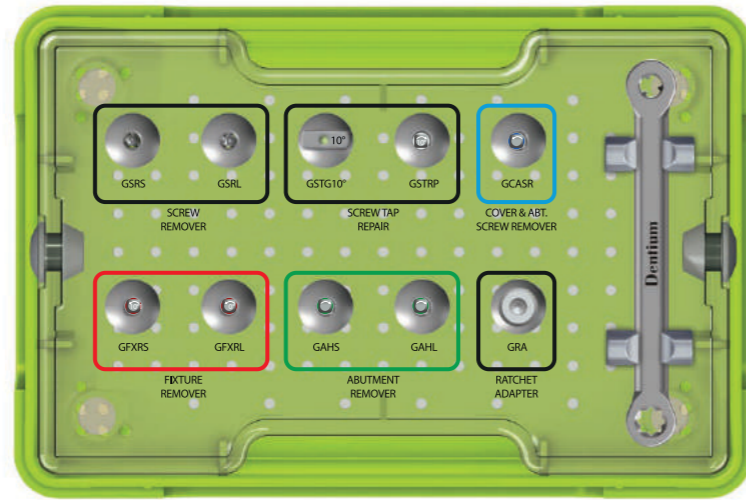

GXIH


## Screw Remover

L	Art. No.
29	GSRS
33	GSRL




## Abutment Remover

L	Art. No.
20	GAHS
25	GAHL

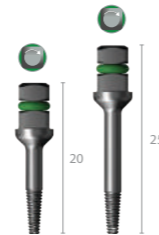



## Screw Tap Repair

L	Art. No.
25	GSTRP
GUIDE	GSTG10

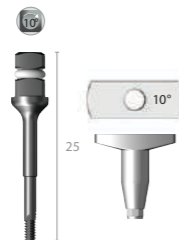


## Fixture Remover

Type	Art. No.
25	GFXRS
30	GFXRL
Wrench	XRFRW

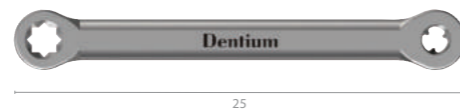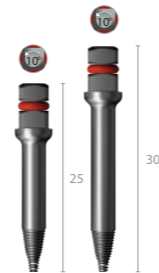

## Cover & Abutment Screw Remover

L	Art. No.
25	GCASR

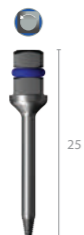

## Ratchet adapter

L	Art. No.
13.9	GRA



## Screw Remover

### Application

To remove the remaining screw when the abutment screw is broken inside the fixture

### Advantage


Easy to remove the broken screw, as well as protect the internal threads of the fixture from being damaged

### Usage

1. Set the torque of the implant motor to 30~50 rpm in a CCW (counterclockwise) direction
2. Assemble the tool with the hand-piece
3. Run the motor while keeping the tip of the tool appropriately contacted with the broken screw until successfully removed


\*Caution: Do not overload the tool with pressure; apply moderate pressure

1

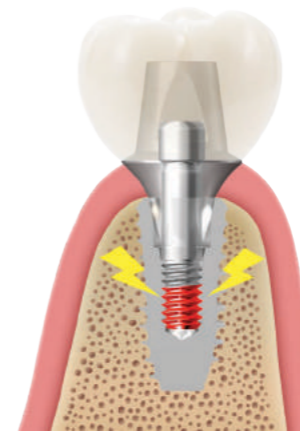

Dual Abutment


2

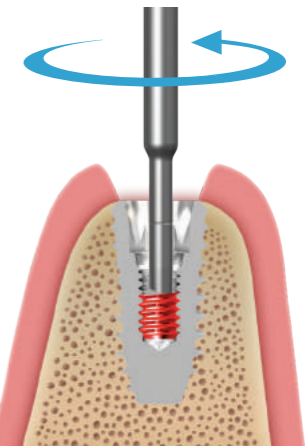

Use the friction force of the tool rotating counterclockwise to remove the screw


Hand-piece Speed: 30~50rpm / CCW

3

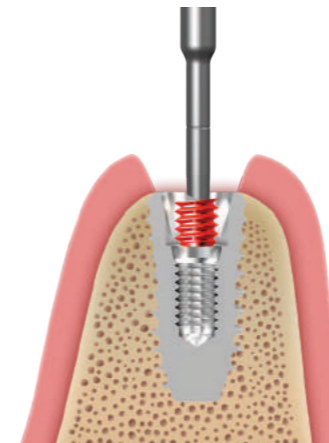

Allow the screw to gradually come out in a swaying motion

4

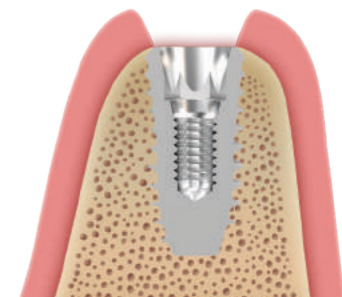

## Abutment Hex Remover

### Application

To remove the remaining hex when the hex portion of an abutment is broken


### Advantage

Easy to remove the broken hex, as well as protect the internal threads of the fixture from being damaged

### Usage

1. Insert the tool inside into the remaining hex hole of the fixture inside
2. Assemble the ratchet with the tool and rotate it in a CW (clockwise) direction to lock the tool tip with the remaining hex
3. Disengage the ratchet and remove the remaining hex by gently rocking the tool
4. If necessary, the hole located in the upper portion of the tool may be used with the crown ejector (not included)

\*Caution: Do not overload the tool with pressure; apply moderate pressure

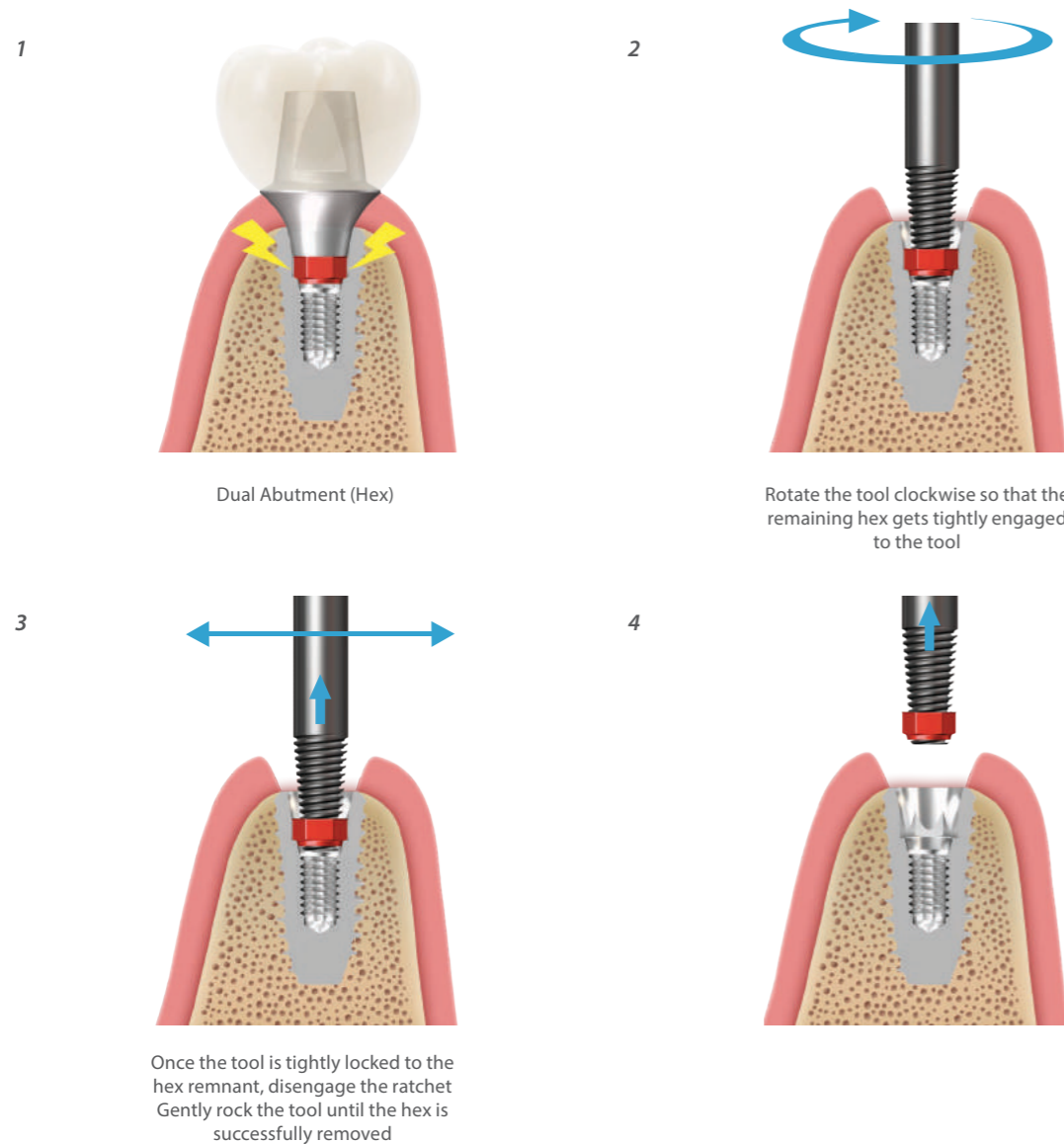

## Screw Tap Repair

### Application

To recreate the internal thread lines of the fixture when it is damaged


### Advantage

Easy to recreate the internal threads with the help of the guides corresponding to different internal angulation (8, 11 degrees) of the fixture

### Usage

1. Place the guide with corresponding degree to the fixture
2. Assemble the tap tool with ratchet
3. Start tapping using the tap tool with appropriate torque
4. If excessive debris accumulates, pause tapping and remove using suction
5. Repeat steps 3 and 4 until completed

\*Caution: Do not apply excessive torque onto the tap tool  
It is highly recommended to use the ratchet after the initial engagement of the tool and the internal threads

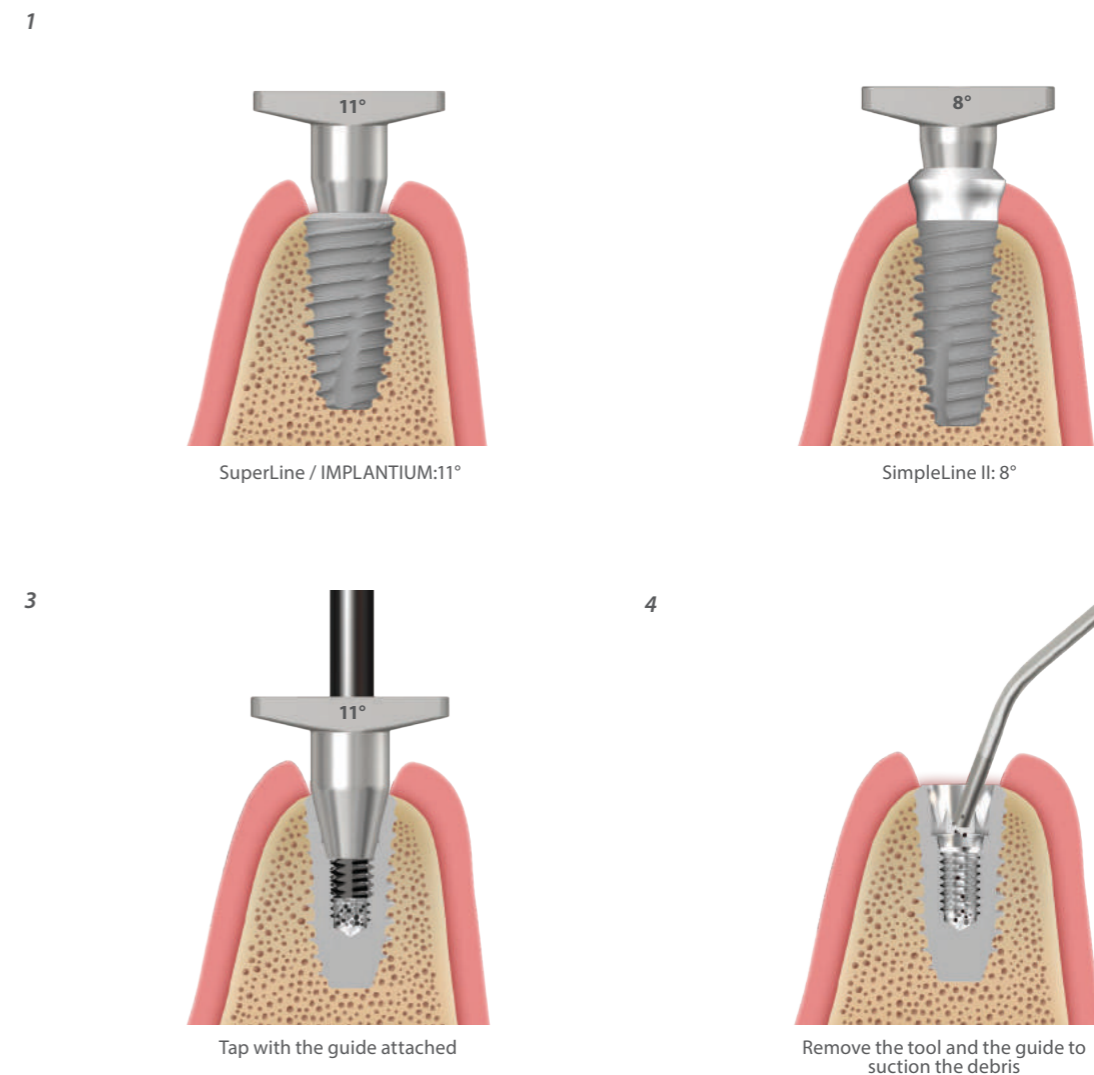

\*If excessive debris accumulates, pause tapping and remove using suction



## Fixture Remover

### Application

To remove the fixture when critically damaged with no other recovery options

### Advantage


Easy to remove the failed fixture without causing damage to the adjacent bone

### Usage

1. Assemble the tool with ratchet, and insert it into the failed fixture to be removed
2. Gently rotate the ratchet in a CCW direction until the tool is tightly locked into the fixture
3. Continue to rotate the ratchet with greater torque in a CCW direction until the failed fixture is completely removed
4. Separate the tool from the removed fixture by rotating it in a CW direction. If necessary, use the wrench (included) to hold the fixture while rotating the tool with ratchet in a CW direction


\*Caution: Sufficient irrigation should be applied to the tool to prevent excessive heating during the procedure

1 Art No. XRFRT

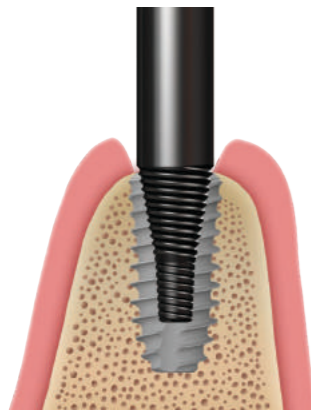


IMPLANTIUM / SuperLine:11°

Art No. XRFRT0

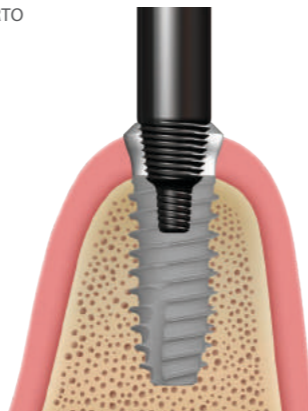

SimpleLine II:8°

2~3

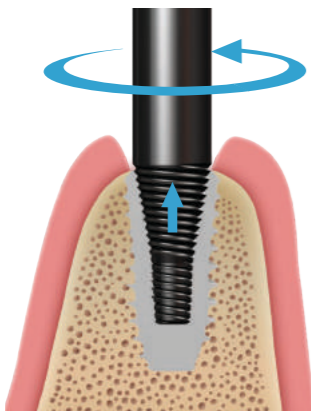

Rotate the tool in a counter clockwise direction until it is tightly locked into the fixture  
Continue to rotate with additional torque until the failed fixture is completely removed

4



Separate the tool from the fixture using the ratchet and the wrench that are included in the kit

## Cover & Abutment Screw Remover

### Application

To disengage the cover screw, healing abutment and abutment screw from the fixture when the 1.28 hex on the head is stripped or damaged


### Advantage

Easy to disengage the cover screw, healing abutment and abutment screw with stripped or damaged hex


### Usage

1. Assemble the tool with the ratchet and place it over the damaged 1.28 hex of the cover screw, healing abutment or abutment screw that needs to be removed
2. Gently rotate the ratchet in a CCW direction to tightly engage the tapered top of the tool into the damaged 1.28 hex.
3. Continue to rotate the ratchet in a CCW direction with greater torque until the cover screw, healing abutment or abutment screw is completely removed
4. After the removal, rotate the ratchet in a CW to separate the tool and the removed component

1 Cover Screw

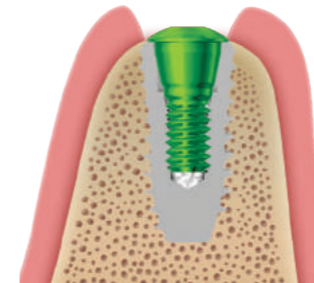


2 Loading downward

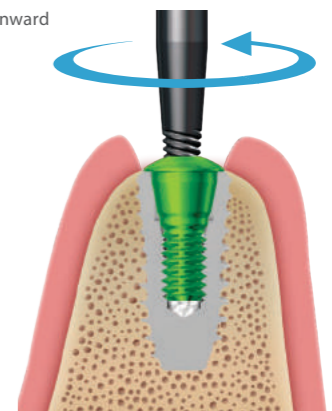


Rotate the tool counterclockwise until tightly locked into the 1.28 hex of the cover screw

3 Abutment Screw

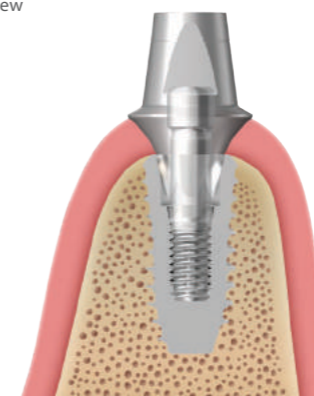

4 Loading downward

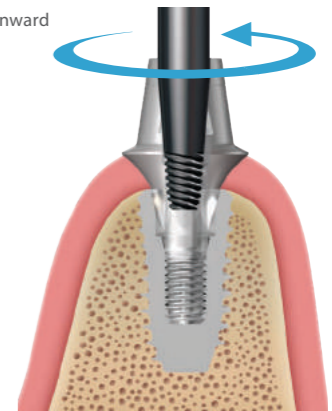


Rotate the tool counterclockwise until tightly locked into the 1.28 hex of the abutment screw

# Temporary Shell

## Preformed Temporary Crown

- Esthetic appearance that mimics a natural tooth
- Convenient for both single and multi-unit restoration

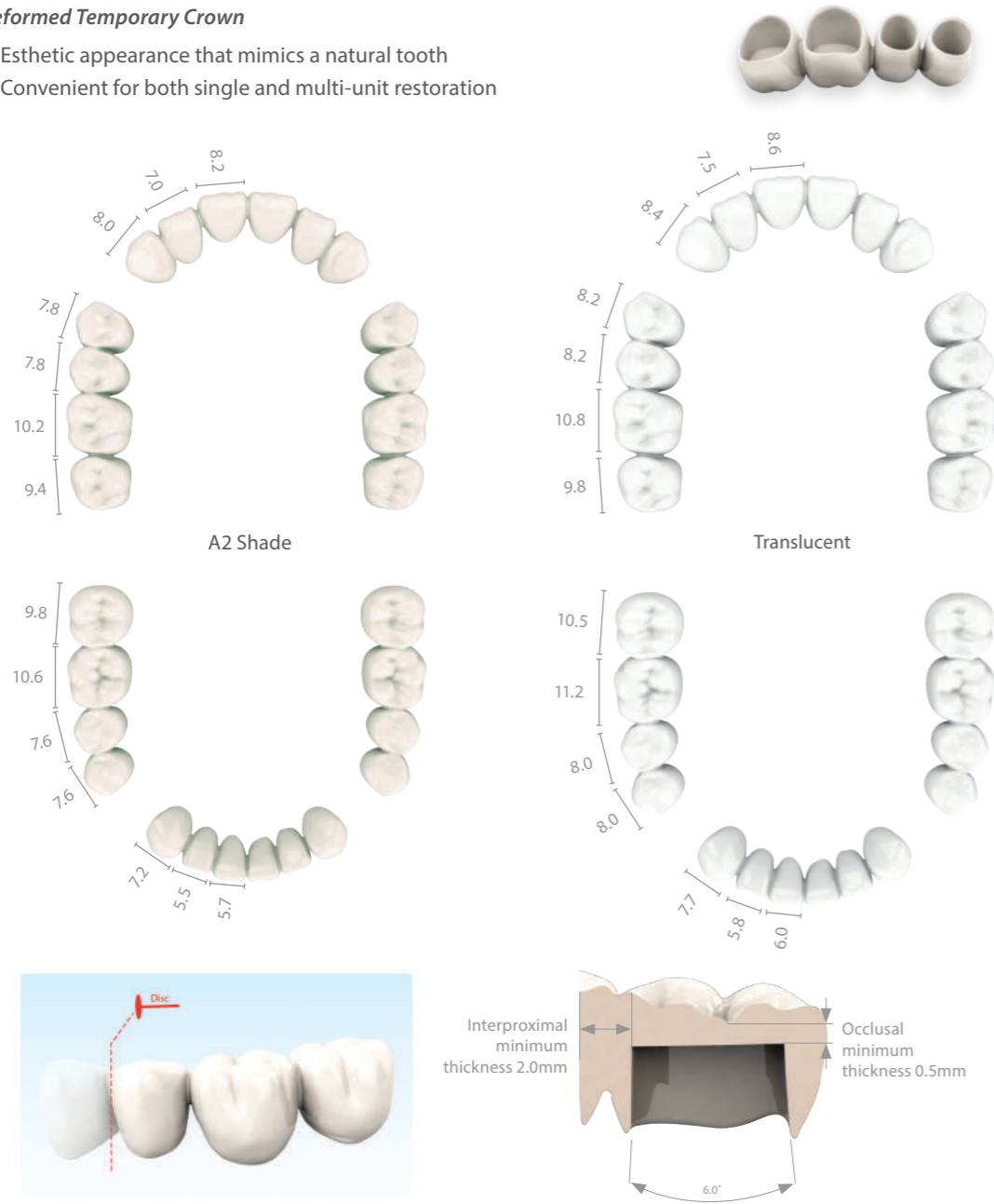

Shade	Type	REF	
	A2	Full set	TSA2-FS
	Translucent	Full set	TSTR-FS



# White Seal

### Easier filling and removal

- Unlike the conventional cotton or impression material, the plush material allows a greater user-friendliness during dental procedures

### No odor & color change

- Odor and color changing problem seen in Silicone or other sealing materials is eliminated

### Stable form maintenance

- The rod with proper stiffness helps maintain its form while preventing the upper application layer from collapsing

### Easy to fill into the screw hole



- White Seal™ is available in 30mm (length) size. The user may easily cut off the desired length and conveniently store away the rest for later use

### Color / Odor Change Test




### Products

Diameter	Length	Art. No.
Ø1.9	30	AHF 19030
Ø2.3	30	AHF 23030

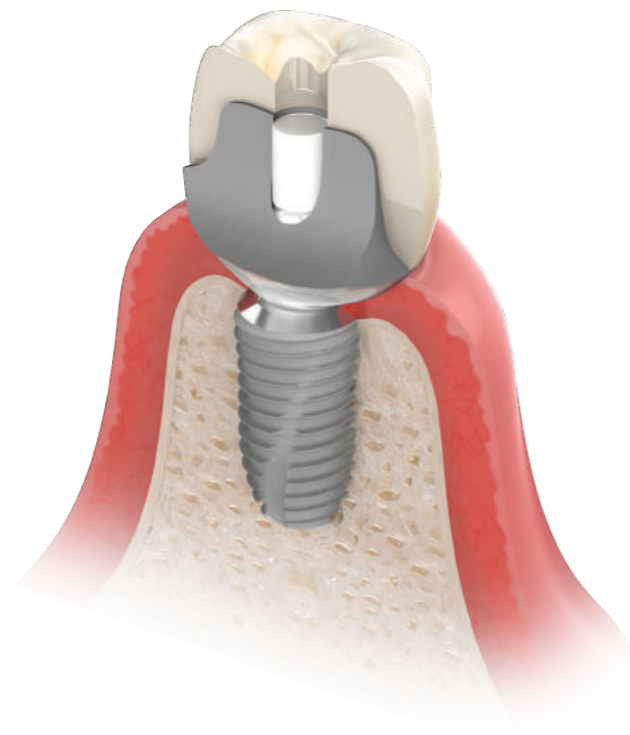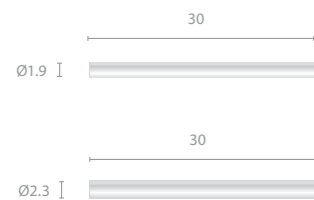

### How to use

Cut off a piece of the White Seal™ in the desired size with scissors or a knife. Insert the piece into the abutment hole and seal it with a resin material. (\*It is recommended to submerge the White Seal™ 2.0mm from the occlusal surface.)

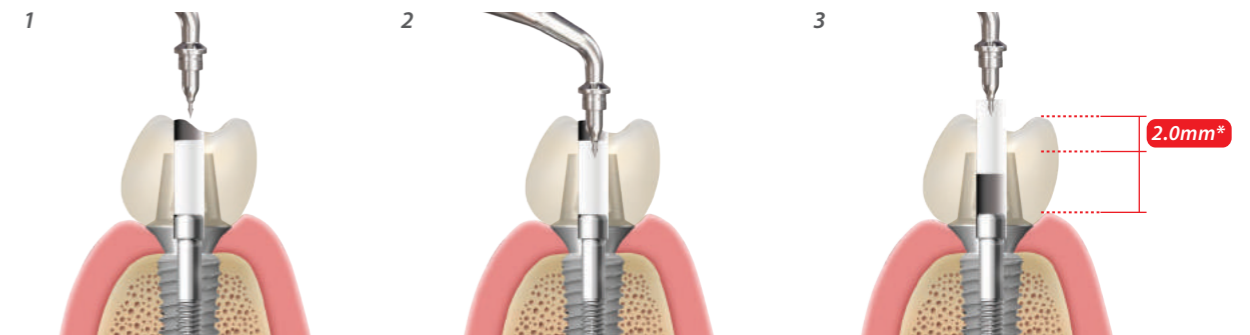

White Seal™

Hole resin filling

After 15 months



# White Seal Tool

## Depth Gauge & Delivery Holder

XSWN



XSWS



Application	Art. No.
NR Line	XSWN
SuperLine, Implantium, SimpleLine II	XSWS

## Remover

XSWR



Application	Art. No.
Use common	XSWR



Delivery Holder




Depth Gauge




Remover


### 1 Cutting the White Seal™

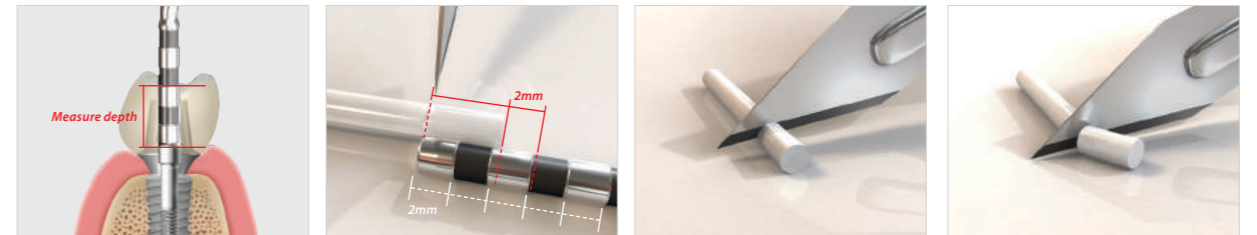


### 2 Delivery the White Seal™ (Delivery holder)

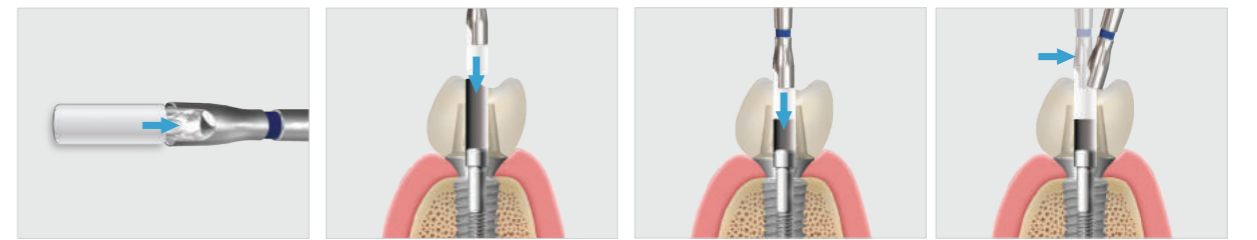

### 3 Delivery the White Seal™ (Depth gauge)

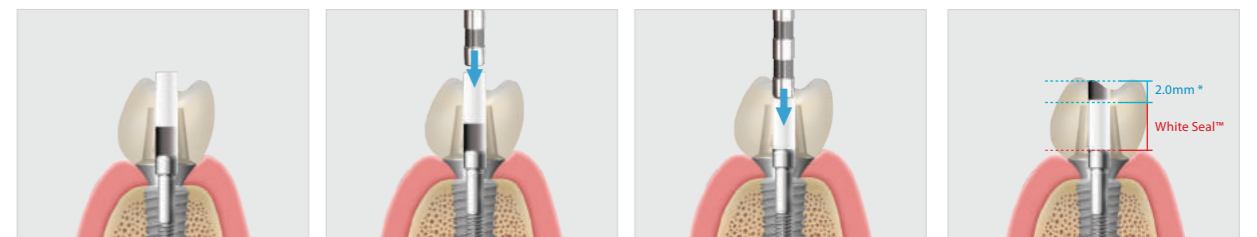


### 4 Removal of the White Seal™ (Remover)

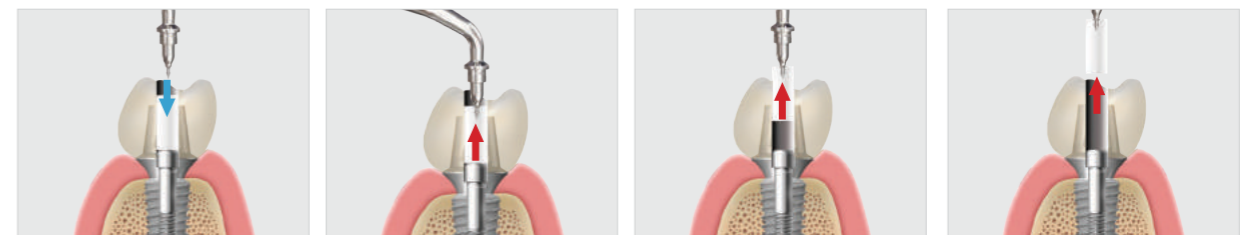

# TN-Brush


- Remove plaque and granulation tissue around the fixture using spinning brush
- The force of shape restoration is excellent with chosen highly elastic brush

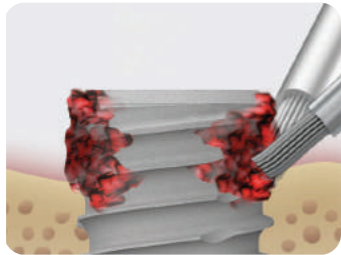

Use brush left to right or top to bottom  
500~800 rpm with irrigation

## Brush


Diameter	Art. No.
Ø0.8	TN-0.8
Ø1.0	TN-1.0
Ø1.3	TN-1.3

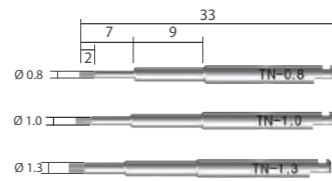


[Unit: mm, Scale 2 : 1]

## Manual


### case 1 \_Peri-implantitis

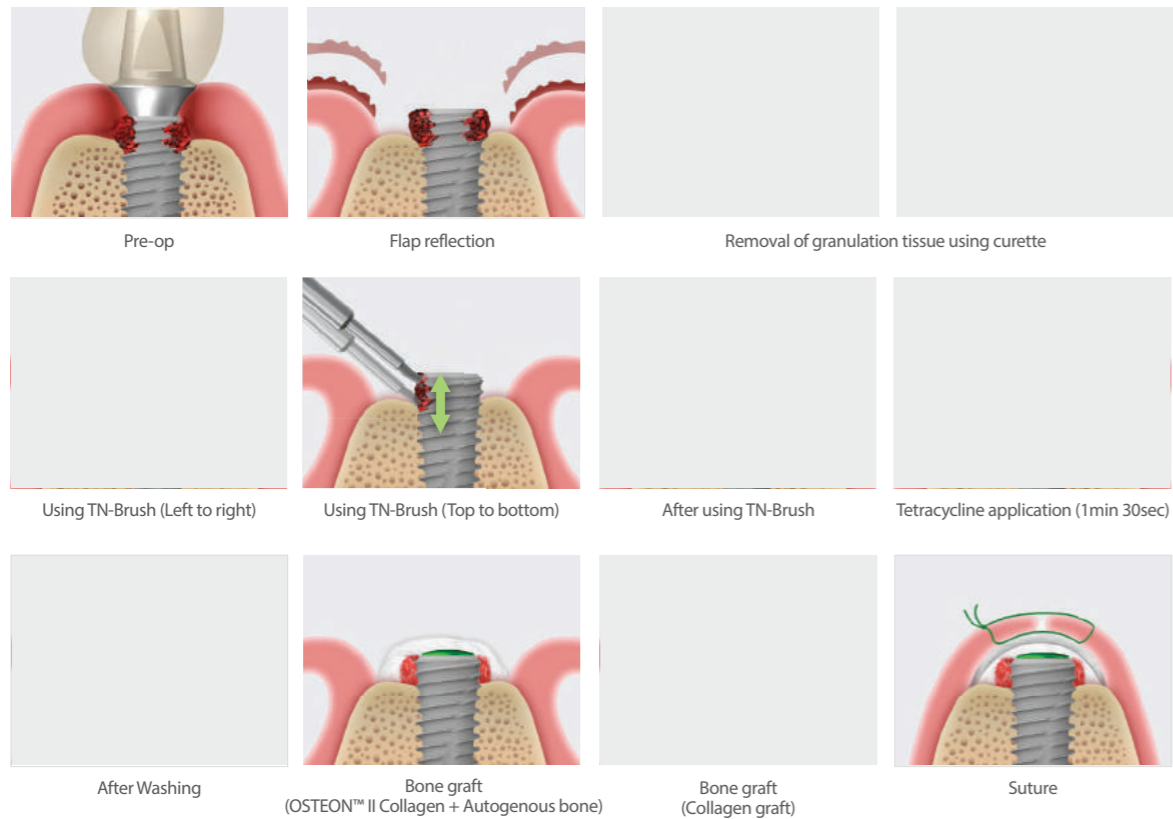

### case 2\_Peri-implantitis

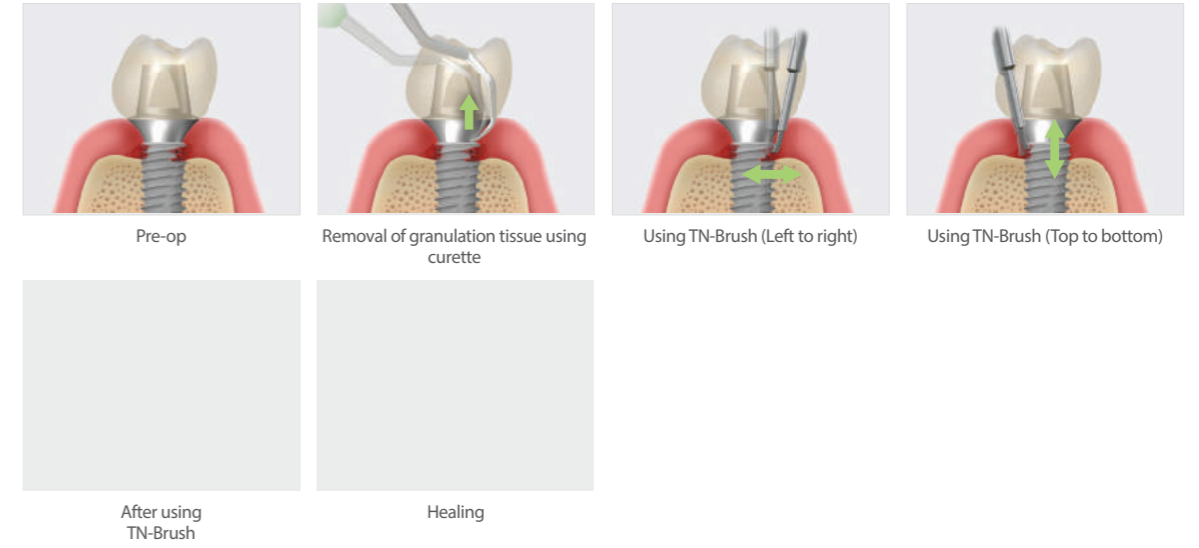


### Case 3\_Peri-implantitis

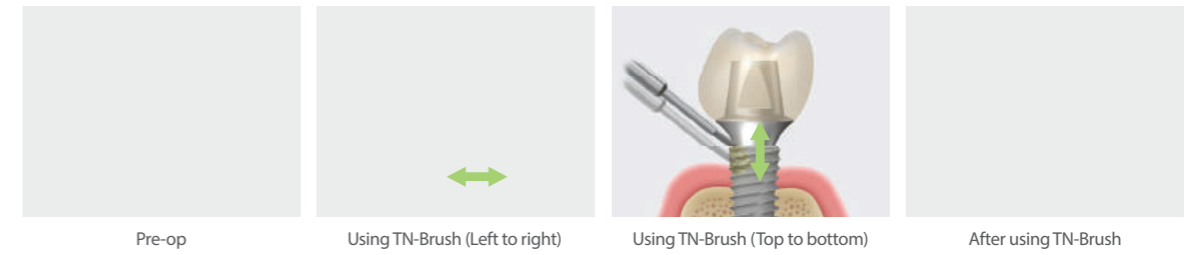


### Case 4\_Periodontal treatment



## SEM

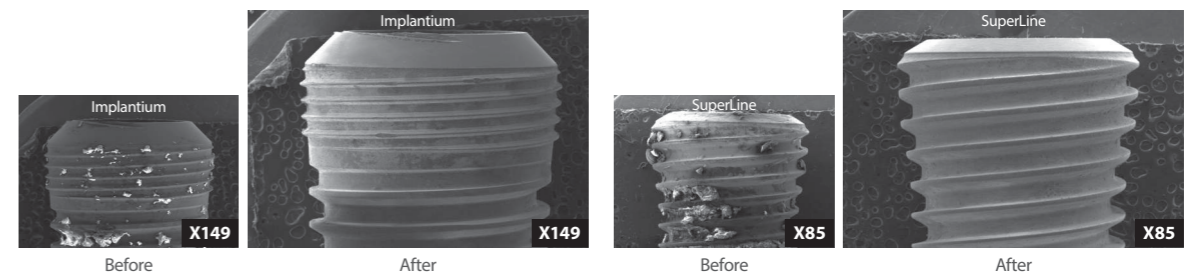




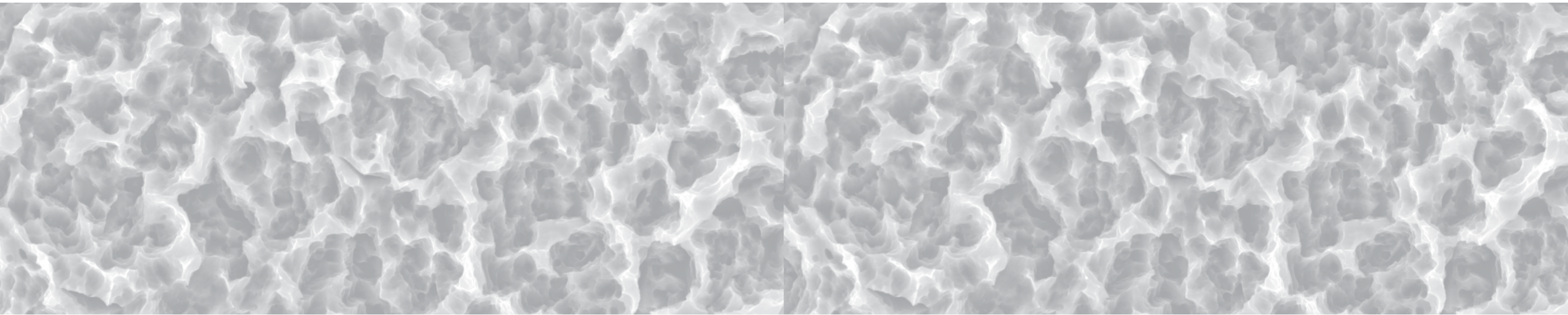

# Simple & Predictable

## 15 Years of Clinical Evidence


S.L.A. (Sandblasting with Large grits and Acid etching) Surface?

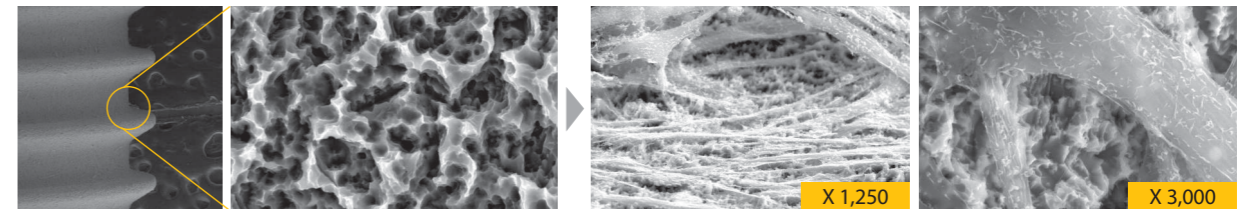

Cell number 3 X 10, after 7 days of cell culture

over  
**15** years  
of Long  
term  
data

OVER A **DECADE** OF COMMITMENT TO THE  
BEST PRODUCTS FOR DENTISTS AND PATIENTS

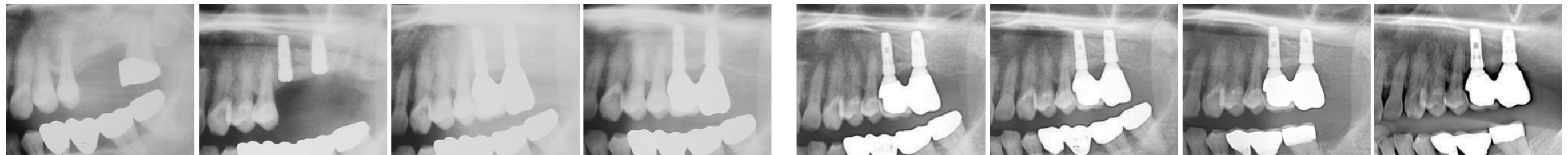

06. 20. 2002  
Pre-op

07. 02. 2002  
Post-op

11. 29. 2002  
Final Prosthesis

06. 02. 2004  
1 Years 2 months

11. 15. 2007  
5 Years

07. 24. 2009  
6 Years 8 months

05. 23. 2012  
9 Years 6 months

11. 25. 2017  
15 Years



# Dentium Instruments

for Total Solution

Catalog & Manual

**Dentium**  
For Dentists By Dentists

Specifications are subject to change without any notice.  
Some products listed in this catalog are not available in the market due to pending approval.